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PLEASE READ THESE INSTRUCTIONS CAREFULLY AND KEEP THEM
Extra-low elasticated corset with flexible stays and elasticated straps

DECLARATION OF CONFORMITY

As the manufacturer, ORTHOSERVICE AG declares, under its sole responsibility, that this is a class I medical device that has been manufactured and assessed according to the EU Regulation 2017/745 (MDR). These instructions have been drawn up in application of the aforementioned Regulation. They are intended to ensure an adequate and safe use of the medical device.

TRADEMARKS OF MATERIALS

Velcro® is a registered trademark of Velcro Industries B.V.
Sensitive® is a registered trademark of EUROJERSEY

SAFETY PRECAUTIONS

We recommend that the pressure exerted by the device does not act on parts of the body with wounds, swelling, or welts. It is advisable to not over tighten the device so as to avoid generating areas of excessive local pressure or the compression of underlying nerves and/or blood vessels. Furthermore, we recommend wearing a garment to avoid direct contact with the skin. If in doubt on how to apply the device, contact a doctor, physiotherapist, or an orthopedic technician. Please read the composition of the product on the inside label carefully; do NOT use the product if the patient has known intolerances or allergies to one or more components in the product. We recommend to not wear the device in the proximity of free flames or strong electromagneticfields. Do not apply in direct contact with open wounds.

WARNINGS

It is advisable that the device, designed for the specific indications as set out below, be prescribed by a doctor or a physiotherapist and applied by an orthopedic technician, according to the specific needs of the patient. To ensure effectiveness, tolerability, and proper functioning, the application must be carried out with the utmost care. Never alter the adjustment made by the doctor/physiotherapist/ or orthopedic technician. The responsibilities of the manufacturer shall lapse in case of inappropriate use or adaptation. The orthosis is made and to be used by only one patient. Should the orthosis be inappropriately used, the manufacturer declines all responsibility, as provided by the regulation for medical devices. In hypersensitive individuals contact with the skin may cause redness and irritation. In case of pain, swelling, welts, or any other adverse reaction, contact your doctor immediately, and in case of a particularly severe adverse event, report the incident to the manufacturer and the competent authority of your country. The effectiveness of the orthopedic product is only guaranteed when all the components are properly in use.

SELECTION/SIZE

Item	REF. 1093
Size	XS S M L XL XXL XXXL *
Pelvic circumf. cm	70/80 80/90 90/100 100/110 110/120 120/130 130/140
Front height cm	14 14 14 14 14 14 14
Back height cm	23 23 23 23 23 23 23
Colour	C:cream / S:black / * XXXL: black only

MAINTENANCE

- ⚠ Do not bleach ⚠ No chemical cleaning
⚠ Do not iron ⚠ Do not tumble-dry
⚠ Washing instructions:
• ensure that the Velcro® fasteners are securely closed before washing the orthosis
• wash the orthosis separately
• wash by hand or in the washing machine with mild detergent using an ultra-gentle cycle at a maximum of 30°C
• leave to dry away from direct heat

Dispose of this product and its parts responsibly

INDICATIONS

- Paravertebral muscle contractures
- Mild traumas
- Mild lumbago
- Mild sciatic lumbalgia and cruralgia
- Post-operative recovery

CONTRAINDICATIONS

Currently no known

CHARACTERISTICS AND MATERIALS

- Structure in elastic Sensitive® fabric, breathable, laminated, rip proof, and antistatic
- Adjustable pulley system with elastic straps for custom fit support
- Velcro® front fastening system
- Spiral stays fixed in an oblique position reinforce the abdominal section
- A soft thermoformed polyethylene foam pad attaches with Velcro® in the lumbar region for added compression
- 4 lightweight and flexible rear stays (width 14mm, height 16cm)

PUTTING ON THE APPLIANCE

- Open the orthosis and temporarily fasten the Velcro® straps on the side (fig. A).
- Put the orthosis in place with the loop on the upper side.
- Insert your thumb in the loop and fasten well the central Velcro® closure (fig. B). Make sure the closure is centered.
- Tighten the orthosis by fastening the straps on the front Velcro® band (fig. C).

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