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**Padded orthosis for stabilising and supporting the dorso-lumbar rachis, with rigid and thermoformable pads**

**DECLARATION OF CONFORMITY**

As the manufacturer, ORTHOSERVICE AG declares, under its sole responsibility, that this is a class I medical device that has been manufactured and assessed, according to the EU Regulation 2017/745 (MDR). These instructions have been drawn up in application of the aforementioned Regulation. They are intended to ensure an adequate and safe use of the medical device.

**TRADEMARKS OF MATERIALS**

Velcro® is a registered trademark of Velcro Industries B.V.

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**WARNINGS**

It is advisable that the device, designed for the specific indications as set out below, be prescribed by a doctor or a physiotherapist and applied by an orthopedic technician, according to the specific needs of the patient. To ensure effectiveness, tolerability, and proper functioning, the application must be carried out with the utmost care. Never alter the adjustment made by the doctor/physiotherapist/ or orthopedic technician. The responsibilities of the manufacturer shall lapse in case of inappropriate use or adaptation.

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**Maintenance**

☒ Do not bleach     No chemical cleaning  
 ☒ Do not iron     Do not tumble-dry  
 ☒ Washing instructions: hand wash at max. 30 °C with neutral soap.  
 Dry away from heat sources.  
 Do not dispose of the product or any of its components into the environment.

**INDICATIONS**

- Back pain due to arthrosis or kyphosis
- After-effects of fractures of dorsal vertebrae and/ or upper lumbar vertebrae
- After-effects of stable compression fractures of the dorsal spine due to osteoporosis or metastasis
- Other indications on medical prescription

**CONTRAINdications**

Currently no known

**CHARACTERISTICS AND MATERIALS**

- Base fabric:  
  - 98% reinforced anelastic cotton for optimal containment
  - Breathable and natural for best comfort
  - Exclusive design
- Anatomical design shaped for the legs. Available in two sizes:  
  - Regular fit (REF. 1335)
  - Tailored fit (REF. 1336)
- Traction system:  
  - Reinforced elastic straps for targeted pressure and perfect stabilization of the lumbar rachis, latex free
  - Pulley system for easy fastening
  - Ferrules in a soft polymer material to facilitate grip and removal of the brace
  - Front Velcro® fastening system, with comfortable grip for easy closing. Shaped for maximum comfort.
  - Padded underarm straps
  - 2 rigid front and back thermoformable pads, on medical prescription
  - Optional rigid stays, moldable on medical prescription, to be inserted in the 4 linear back pockets.
  - Adapts to the pathology of the patient.

**PUTTING ON THE APPLIANCE**

- 1 Open up the orthosis and provisionally affix the Velcro® straps to the strips on the side (fig. A). Position the support in such a way that the inside label faces upwards.
- 2 Put on the orthosis by sliding the arms through the shoulder straps as if it were a backpack (fig. B).
- 3 Use the middle Velcro® closure to fix in place securely (fig. C). Ensure that the closure is precisely centred.
- 4 Slowly tighten up the support by securing the lower Velcro® straps to the front strips (fig. D).
- 5 Then secure the upper straps to the front Velcro® strips (fig. E).
- 6 Grasp the straps and pull towards the front (fig. F). Attach them to the Velcro® on the strip at the front (fig. G).

**To apply the rear rigid pad (heat-mouldable):**

- apply the adhesive padding to the plastic part (concave side-without any writing, fig.H)
- apply the adhesive Velcro® to the sides of the central rib (fig.I)
- attach the pad with Velcro® to the internal dorsal part of the orthosis (fig.L)

**To apply the front rigid pad (heat-mouldable):**

- apply the adhesive padding to the plastic part (concave side-without any writing)
- attach the pad with Velcro® to the front closure of the orthosis

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