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PLEASE READ THESE INSTRUCTIONS CAREFULLY AND KEEP THEM

Spinal brace for osteoporosis DECLARATION OF CONFORMITY As the manufacturer, ORTHOSERVICE AG declares, under its sole responsibility, that this is a class I medical device that has been manufactured and assessed, according to the EU Regulation 2017/745 (MDR).

TRADEMARKS OF MATERIALS Velcro® is a registered trademark of Velcro Industries B.V. SAFETY PRECAUTIONS We recommend that the pressure exerted by the device does not act on parts of the body with wounds, swelling, or welts.

WARNINGS It is advisable that the device, designed for the specific indications as set out below, be prescribed by a doctor or a physiotherapist and applied by an orthopedic technician, according to the specific needs of the patient.

Table with 5 columns: Item, REF.33049, Size (XS, S, M, L), Distance T1-L5 cm, Pelvic circ. cm, Colour (grey)

REF. 33049/P optional inflatable pad Abdominal and dorsal tensioning straps, made from comfortable belt material

MAINTENANCE Do not bleach, Do not iron, Do not tumble-dry, Washing instructions: Pull out the metal frame, see points 1-a, b, c. Wash by hand in lukewarm water with neutral soap, rinse thoroughly.

INDICATIONS Dorsolumbalgia after healing of stable compression fractures of dorsolumbar vertebrae Stable compression fractures of dorsolumbar vertebrae caused by osteoporosis or metastatic tumors

CHARACTERISTICS AND MATERIALS Malleable radiotranslucent aluminium alloy inner frame Patented strap system Fastens around the abdomen with a corset

ADAPTING FOR THE DOCTOR/TECHNICIAN 1 Remove the metal frame from the padding: Unfasten the four press studs (fig. A) that attach the pad to the metal frame.

2 To shorten the corset: Undo the Y-shaped fastenings and remove the abdominal corset from the dorsal part (fig. K1). Measure the circumference of the patient's pelvis.

Table with 5 columns: Pelvis circumference cm, Minimum amount to cut off each of the two sides cm, 70, 80, 90, 100, 110, 25, 20, 15, 10, 5

3 To put on the brace, slide the patient's arms through the shoulder straps (fig. J). 4 Fasten the abdominal pads with Velcro® so that they are centred on the abdomen in a slightly staggered position (fig. I).

PUTTING ON THE APPLIANCE FOR THE PATIENT 1 Fit the orthosis by inserting the arms in the shoulder straps (fig. 1). 2 Close the abdominal pelottes using the Velcro® closure. Ensure that they are centred (fig. 2).

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TRADEMARKS OF MATERIALS Velcro® est une marque enregistrée de Velcro Industries B.V. PRÉCAUTIONS D'USAGE Veillez à ce que la pression exercée par le dispositif n'agisse pas sur les parties du corps présentant des lésions, enflures ou tuméfactions.