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   
 **PLEASE READ THE INSTRUCTIONS CAREFULLY AND THOROUGHLY AND KEEP THEM FOR FUTURE REFERENCE**

**Elbow support in AirX™ fabric**

**DECLARATION OF CONFORMITY**

As the manufacturer, ORTHOSERVICE AG states under its sole responsibility that this product is a Class I medical device that has been manufactured in accordance with the requirements of Directives 93/42/EEC and 2007/47/EC. These instructions have been written in conformity with these directives, and their aim is to ensure the safe and proper use of this medical device.

**TRADEMARKS OF MATERIALS**

Velcro® is a registered trademark of Velcro Industries B.V.

**PRECAUTIONS**

It is advised that the pressure exerted by the product does not act on parts of the body suffering from wounds, swelling or lumps. The product should not be tightened too firmly so as to avoid the development of pressure points or compression of the nerves or blood vessels underneath. If in doubt about how to apply, contact an orthopaedic technician. Please read the composition of the product on the inside label carefully. The device should not be worn in the vicinity of open flames.

**WARNINGS**

The product has been designed for the specific indications stated below and must be prescribed by a doctor and fitted by an orthopaedic technician. The orthopaedic technician is responsible for fitting the product and providing information about how to use it safely in accordance with individual requirements. The product must be fitted with the utmost care to ensure its efficacy, tolerability and correct function. Any changes to the structure or adjustment of the device must be prescribed by a physician and performed by an orthopaedic technician. Under no circumstances should the adjustment made by the physician/orthopaedic technician be altered. The manufacturer is no longer liable if the product is used or adapted inappropriately. The orthosis is intended to be used only by a single patient; if it is used for more than one patient, the manufacturer accepts no responsibility or liability in accordance with medical device legislation. Direct contact with the skin may cause reddening or irritation in hypersensitive individuals. In the case of onset of pain, swelling, lumps or any unusual reaction, immediately contact your physician. The orthopaedic efficacy of the product may only be guaranteed when all of its components are used.

**SELECTION/SIZE**

Item	REF.B1132				
Size	XS	S	M	L	XL
Proximal circumf. of forearm cm	20/23	23/25	25/30	30/35	35/40
Colour	black				
fits right and left					

**MAINTENANCE**

⚠ Do not bleach    ⛔ No chemical cleaning

⚠ Do not iron    ⛔ Do not tumble-dry

🧼 Washing instructions: Hand wash in lukewarm water (30°C) with mild soap; rinse thoroughly. Leave to dry away from direct heat.

Dispose of this product and its parts responsibly

**INDICATIONS**

- Epicondylitis and epitrochleitis of the elbow
- Chronic pain from bursitis or tendinitis

**CONTRAINDICATIONS**

Currently no known

**CHARACTERISTICS AND MATERIALS**

- Made from breathable AirX™ material
- Soft-X insert for maximum comfort
- Velcro® strap with nylon fastener

**PUTTING ON THE APPLIANCE**

- Open up the Velcro® strap and close loosely and provisionally.
- Pull the elbow support over the arm so that the orange label faces upwards and the SoftX insert is located on the inner arm in the bend of the elbow; the Velcro® strap must be located just under the epicondyle (fig. A).
- Tighten the strap until the desired pressure is achieved and secure using the Velcro® closure (fig. B and C).

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Velcro® è un marchio di fabbrica registrato di Velcro Industries B.V.