REF.725

REF.726

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PLEASE READ THESE INSTRUCTIONS CAREFULLY AND KEEP THEM

REF.725 Humeral fracture brace
REF.726 Shoulder cap humeral brace

DECLARATION OF CONFORMITY

ORTHOSERVICE AG declares, under its sole responsibility, that this is a class I medical device that has been manufactured and assessed, according to the EU Regulation 2017/745 (MDR).

TRADEMARKS OF MATERIALS

Velcro® is a registered trademark of Velcro Industries B.V.

SAFETY PRECAUTIONS

We recommend that the pressure exerted by the device does not act on parts of the body with wounds, swelling, or welts. It is advisable to not over tighten the device so as to avoid generating areas of excessive local pressure or the compression of underlying nerves and/or blood vessels.

Furthermore, we recommend wearing a garment to avoid direct contact with the skin.

If in doubt on how to apply the device, contact a doctor, physiotherapist, or an orthopedic technician. Read the product composition on the internal label carefully.

We recommend to not wear the device in the proximity of free flames.

Do not apply in direct contact with open wounds.

WARNINGS

It is advisable that the device, designed for the specific indications as set out below, be prescribed by a doctor or a physiotherapist and applied by an orthopedic technician, according to the specific needs of the patient.

To ensure effectiveness, tolerability, and proper functioning, the application must be carried out with the utmost care. Never alter the adjustment made by the doctor/physiotherapist/ or orthopedic technician.

In hypersensitive individuals, direct contact with the skin may cause redness and irritation. In case of pain, swelling, welts, or any other adverse reaction, contact your doctor immediately, and in case of a particularly severe adverse event, report the incident to the manufacturer and the competent authority of your country.

The effectiveness of the orthopedic product is only guaranteed when all the components are properly in use.

SELECTION/SIZE

Table with columns for Code, Size, and measurements for XS, S, M, L, XL for REF.725 and REF.726.

Fits right and left
\* you may shorten by cutting it

MAINTENANCE

- ☒ Do not bleach ☒ No chemical cleaning
☒ Do not iron ☒ Do not tumble-dry
☒ Instructions for washing: wipe with a sponge dipped in lukewarm water and mild soap. Dry with a cloth.

INDICATIONS

- Stable fractures of medial distal third of humerus
• Post-surgery brace after humerus fractures
• Fractures due to non-operative bone metastasis (Only REF. 726)

CONTRAINDICATIONS

Currently no known

CHARACTERISTICS AND MATERIALS

- Structure made of Polyethylene (Only REF. 725)
• Structure made of Polyethylene (Only REF. 726)
• Internal pad made of close cells against moisture
• Extension cap to shoulder-humeral joint for better control of movements (Only REF. 726)
• Velcro® straps
• Ventilation holes
• Two tubular sleeves included

PUTTING ON THE APPLIANCE REF.725

- 1 Slip the protective sock onto the upper arm.
2 Don the brace and tighten the Velcro® straps. The open side must correspond with the inner side of the arm.
3 Don the arm support belt.

PUTTING ON THE APPLIANCE REF.726

- 1 If necessary, mould the plastic parts with the help of a heat gun.
2 Slip the protective sock onto the upper arm.
3 Don the brace and tighten the Velcro® straps.
4 Fasten the shoulder cap securely with the Velcro® chest strap.

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Table with columns for Code, Taille, and measurements for XS, S, M, L, XL for REF.725 and REF.726.

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