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**PLEASE READ THESE INSTRUCTIONS CAREFULLY AND KEEP THEM**

**REF.G1108 - G1108A AirX™ Knee brace**  
**REF.G1115 - G1115A AirX™ Knee orthosis with hinges**

**DECLARATION OF CONFORMITY**  
As the manufacturer, ORTHOSERVICE AG declares, under its sole responsibility, that this is a class I medical device that has been medically designed and assessed, according to the EU Regulation 2017/745 (MDR). These instructions have been drawn up in application of the aforementioned Regulation. They are intended to ensure an adequate and safe use of the medical device.

**TRADEMARKS OF MATERIALS**  
Velcro® is a registered trademark of Velcro Industries B.V.

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**WARNINGS**  
It is advisable that the device, designed for the specific indications as set out below, be prescribed by a doctor or a physiotherapist and applied by an orthopedic technician, according to the specific needs of the patient. To ensure effectiveness, tolerability, and proper functioning, the application must be carried out with the utmost care. Never alter the adjustment made by the doctor/physiotherapist or orthopedic technician. The responsibilities of the manufacturer shall lapse in case of inappropriate use or adaptation. The orthosis is made and to be used by only one patient. Should the orthosis be inappropriately used, the manufacturer declines all responsibility, as provided by the regulation for medical devices. In hypersensitive individuals, direct contact with the skin may cause redness and irritation. In case of pain, swelling, welts, or any other adverse reaction, contact your doctor immediately, and in case of a particularly severe adverse event, report the incident to the manufacturer and the competent authority of your country. The effectiveness of the orthopedic product is only guaranteed when all the components are properly in use.

Code	REF.G1108 - G1108A / REF.G1115 - G1115A					
Size	XS	S	M	L	XL	XXL
Thigh circ. taken 15 cm above mid-patella cm	33/39	39/45	45/52	52/58	58/66	66/75
Length cm	34	34	34	34	34	34
Colour	black					

fits right and left

**MAINTENANCE**  
 Do not bleach  No chemical cleaning  
 Do not tumble-dry  Do not iron  
 Washing instructions: hand wash at max. 30° C with neutral soap, (REF. G1115 - G1115A only: after removing the hinge bars), do not dry in the vicinity of heat sources.  
 Do not dispose of the product or any of its components into the environment.

**INDICATIONS**  
 • REF. G1108 - G1108A only: Early stage knee pain or effects after sprain trauma with mild residual collateral ligament laxity (non-operative knee)  
 • REF. G1115 - G1115A only: Knee pain or effects after sprain trauma with mild residual collateral ligament laxity (non-operative knee)  
 • REF. G1108 - G1115: Mild mid-lateral patella instability  
 • REF. G1108 - G1115 only: Patellofemoral arthrosis  
**KONTRAINDICATIONS**  
 Currently no known

**CHARACTERISTICS AND MATERIALS**  
 • New anatomical, ultra-flat design for a flexible movement and perfect adherence  
 • Breathable AirX™ fabric  
 • New softer Soft-X fabric for kneepit insert (with no central stitch) for better comfort  
 • Softer and elastic edge  
 • Genufit 08/15 only: Patella foam stabilizer  
 • Genufit 08A/15A only: New double front opening to apply the brace with ease: internal fastening system for prefixing and external fastening system for final fixing. Patella cut-out  
 • New fastening system: straps with buckles and elastic support for adjusting pressure  
 • Genufit 08/08A only: New low profile pockets made of flexible foam, stay and strap covers  
 • Genufit 08/08A only: Supporting lateral spiral stays  
 • Genufit 15/15A only: Ultra low-profile polycentric hinges

**PUTTING ON THE APPLIANCE**  
**ADAPTING FOR THE DOCTOR/TECHNICIAN**  
 1 Open all Velcro® closures and provisionally close them on themselves.  
 2 **Genufit 08 - Genufit 15**  
 Pull the knee brace over the knee so that the orange label is facing upwards (fig. A) and the patella cut-out is located above the knee cap (fig. B); the guides/joint braces should be parallel to one another.  
**Genufit 08A - Genufit 15A**  
 Pull the knee brace on in such a way that the orange label is facing upwards and the patella cut-out is located over the knee cap; then fasten the inner Velcro® closures (fig. C and D) and the outer Velcro® closures (fig. E and F).  
 3 **Genufit 15 - Genufit 15A**  
 If necessary, remove the knee brace, take the joint braces out of teh pockets and bend to correspond to the shape of the leg (fig. G). Then replace the joint braces back in the pockets and put the knee brace back on again.  
 4 Pull the two Y-shaped ends of the upper strap through the appropriate strap guide, pull it firmly, simultaneously with both hands and then secure the Velcro® closure (fig. H). Repeat this process for the lower strap (fig. I).  
 5 If necessary, the straps may be shortened:  
 • Remove the Y-shaped Velcro® closures at the end of the strap (fig. L)  
 • Cut the strap to the appropriate length (fig. M)  
 • Attach the Y-shaped Velcro® closures again (fig. L)

**PUTTING ON THE APPLIANCE FOR THE PATIENT**  
 1 Open all Velcro® closures and provisionally close them on themselves.  
 2 **Genufit 08 - Genufit 15**  
 Pull the knee brace over the knee so that the orange label is facing upwards (fig. A) and the patella cut-out is located above the knee cap (fig. B); the guides/joint braces should be parallel to one another.  
**Genufit 08A - Genufit 15A**  
 Pull the knee brace on in such a way that the orange label is facing upwards and the patella cut-out is located over the knee cap; then fasten the inner Velcro® closures (fig. C and D) and the outer Velcro® closures (fig. E and F).  
 3 Pull the two Y-shaped ends of the upper strap through the appropriate strap guide, pull it firmly, simultaneously with both hands and then secure the Velcro® closure (fig. H). Repeat this process for the lower strap (fig. I).

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