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EN **FR** **IT** **PLEASE READ THE INSTRUCTIONS CAREFULLY AND THOROUGHLY AND KEEP THEM FOR FUTURE REFERENCE**

Knee orthosis with polycentric articulated hinges with F-E control

DECLARATION OF CONFORMITY

As the manufacturer, ORTHOSERVICE AG states under its sole responsibility that this product is a Class I medical device that has been manufactured in accordance with the requirements of Directives 93/42/EEC and 2007/47/EC. These instructions have been written in conformity with these directives, and their aim is to ensure the safe and proper use of this medical device.

TRADEMARKS OF MATERIALS

Velcro® is a registered trademark of Velcro Industries B.V.

PRECAUTIONS

It is advised that the pressure exerted by the product does not act on parts of the body suffering from wounds, swelling or bruising. The product should not be tightened too firmly so as to avoid the development of pressure points or compression of the nerves or blood vessels underneath. If in doubt about how to apply, contact an orthopaedic technician. Please read the composition of the product on the inside label carefully. The device should not be worn in the vicinity of open flames or strong electromagnetic fields.

WARNINGS

The product has been designed for the specific indications stated below and must be prescribed by a doctor and fitted by an orthopaedic technician. The orthopaedic technician is responsible for fitting the product and providing information (a) conforming with any exigencies of medical device legislation, and (b) in accordance with individual requirements. The product must be fitted with the utmost care to ensure its efficacy, tolerability and correct function. Any changes to the structure or adjustment of the device must be prescribed by a physician and performed by an orthopaedic technician. Under no circumstances should the adjustment made by the physician/orthopaedic technician be altered. The manufacturer is no longer liable if the product is used or adapted inappropriately. The orthosis is intended to be used only by a single patient; if it is used for more than one patient, the manufacturer accepts no responsibility or liability in accordance with medical device legislation. Direct contact with the skin may cause reddening or irritation in hypersensitive individuals. In the case of onset of pain, swelling, lumps or any unusual reaction, immediately contact your physician. The orthopaedic efficacy of the product may only be guaranteed when all of its components are used.

WARRANTY

This warranty is governed by the statutory provisions of the country in which the product was purchased. Where relevant that country's regulations concerning warranties and the relationship between the seller and buyer shall apply. If you are not satisfied with the quality of the warranty, first contact the retailer where the product was purchased. Orthoservice will repair or replace all or part of the device and its accessories if defects in materials and workmanship arise during the period of six months from the date of purchase.

SELECTION/SIZE

Code	REF. 33122	33122A	REF. 33126	33126A	REF. 33124A	33128A
Size	XS	S	M	L	XL	XXL
Thigh circ. taken 15 cm above mid-patella cm	33/39	39/45	45/52	52/58	58/66	66/75
Length cm	34	34	34	34	34	34
Genuskill 22,22A,26,26A	41	41	41	41	41	41
Genuskill 24A,28A						
Colour	grey					

fits right and left

MAINTENANCE

- Do not bleach
- No chemical cleaning
- Do not iron
- Do not tumble-dry
- Washing instructions: hand wash at max. 30° C with neutral soap. Dry away from heat sources.
- Dispose of this product and its parts responsibly

INDICATIONS

- Functional treatment of sprain injuries grade I and II of the collateral medial/lateral ligament
- Prevention of re-injury of MCL and LCL sprains during sports activities
- Chronic latero-medial instability
- Femoro-patellar arthrosis
- Arthritic instability (inoperable knee)

CONTRAINDICATIONS

Currently no known

CHARACTERISTICS AND MATERIALS

- **BreathSkill** fabric support:
 - soft and breathable
 - 3D weave
 - a special woven mesh covering for a stronger, more durable design
- **CompresSkill** on the back of the support:
 - Stretchy multi-fibre yarn for compression
 - Ultra multi-fibre elastic and compressif
 - Fully adaptable
 - Perfect snug fit around the jointo
- Only GenusSkill22,26: FlexSkill on the front of the support (bonded with a technical fabric) with special low-profile bonded seams. Patella stabiliser included.
- Only GenusSkill22A, 24A, 26A, 28A: Front Velcro® fastening; the new and improved design makes fastening even easier.
- Solo GenusSkill22,22A,24A: The polycentric hinge can be easily adjusted with pins on the exterior. Set the flexion, extension, or lock at 0°, 15°, 30°, 60°, or 90°
- Only GenusSkill26,26A,28A: the new polycentric CLICKKLACK hinge can be easily adjusted using wedges and WITHOUT the need for any tools:
 - Extension: 0°, 10°, 20°, 30°, 40°
 - Flexion: 45°, 60°, 75°, 90°
 - Lock: 0°, 10°, 20°, 30°
- A new strap fastening system:
 - Y-shaped Velcro® fastenings for the initial adjustments
 - Subsequent fastening and unfastening using a rapid-release buckle, ring-and-loop system, and Velcro® fastenings

ADAPTING FOR THE DOCTOR/TECHNICIAN

- 1 The knee support is supplied with the straps undone so it is ready to be adjusted.
 - GenusSkill22 - GenusSkill26**
 - Slide the knee support onto the leg with the orange label at the top (fig. A), and centre the front opening over the kneecap.
 - GenusSkill22A - GenusSkill24A - GenusSkill26A - GenusSkill28A**
 - Open the knee support, position it with the orange label at the top, wrap it around the extended knee, and centre the kneecap in the opening provided as best as possible. Fasten the Velcro® straps (fasten the top one first (fig. B), and then the lower one (fig. C)).
- 2 If necessary, remove the knee support, remove the stays from their pockets, shape as required, slide the stays back into their pockets, and don the support once again.
- 3 Adjust the straps (on the long supports: start with the distal thigh strap):
 - Pull the strap to the front
 - Secure it to the Velcro® fastening above the stay opposite the buckle (fig. D)
 - Slide the tip through the corresponding ring at the back and fasten it back on itself using the Velcro® (fig. E). If it is too long, it can be cut to size and the corresponding Y-shaped Velcro® can be reattached.
- 4 Set the required degree of flexion and extension if necessary.
 - GenusSkill22 - GenusSkill22A - GenusSkill24A**
 - Open the cover (fig. F), set the pins to the required degree of flexion/ extension (fig. G) and close the cover.
 - GenusSkill26 - GenusSkill26A - GenusSkill28A**
 - Open the hinge cover by sliding the two half-covers to the side (fig. H), remove the central locking component (fig. I) and insert/replace the wedges to the required degree of flexion/extension (fig. L)
 - Replace the central locking component (fig. M) and slide the two halfcovers towards the centre until they securely "click" into place (fig. N)

PUTTING ON THE APPLIANCE FOR THE PATIENT

- 1 Undo all the straps:
 - Undo the Velcro® fastening (fig. O)
 - Undo the buckle (fig. P)
 - For the closed models (GenusSkill22 - GenusSkill26) unfasten the strap from the Velcro® fastening above the stay opposite the buckle as well
- 2 Don the knee support
 - GenusSkill22 - GenusSkill26**
 - Slide the knee support onto the leg with the orange label at the top (fig. Q). Centre the front opening over the kneecap.
 - GenusSkill22A - GenusSkill24A - GenusSkill26A - GenusSkill28A**
 - Open the knee support, position it with the orange label at the top, wrap it around the extended knee, and centre the kneecap in the opening provided as best as possible. Fasten the Velcro® straps, fasten the top one first (fig. R), and then the lower one (fig. S).
- 3 Fasten the straps (on the long supports: start with the distal thigh strap):
 - For the closed models (GenusSkill22 - GenusSkill26) tighten the strap and secure it to the Velcro® fastening above the stay opposite the buckle (fig. T)
 - Fasten the buckle (fig. U)
 - Secure the corresponding Velcro® fastening (fig. V)

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