


# A debilitating disease with a substantial social impact

Increasingly more people suffer from gonarthrosis. It affects the old and young alike, especially if they practise high-intensity sports, do a physically demanding job, or have experienced previous knee injuries. This disease often involves taking sizeable quantities of pain-relief medication.

We are committed to alleviating the pain and discomfort felt by so many patients which is why we originally designed the **ORTHO-A** knee brace for gonarthrosis.

## APPLICATIONS

In the case of unicompartmental gonarthrosis, the **ORTHO-A** knee brace uses targeted pressure to help realign the joint, increasing the separation between the femoral condyle and the tibial condyle and "virtually" rebuilding the space previously occupied by the cartilage. With less weight now on the compromised compartment (medial or lateral), patients experience considerably less pain and stiffness.

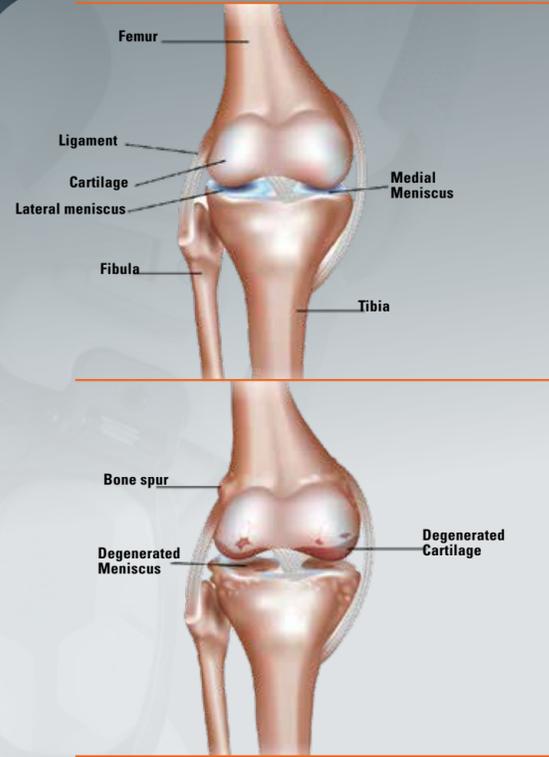

90CDP00089EN02\_06Z1

## ORTHO-A

Orthosis for gonarthrosis

Item	REF.91014 right or left
Size	one size
Colour	grey

right or left

**RIGHT-** for medial gonarthrosis of the right knee (varus right knee) or for lateral gonarthrosis of the left knee (valgus left knee).  
**LEFT-** for medial gonarthrosis of the left knee (varus left knee) or for lateral gonarthrosis of the right knee (valgus right knee).

### Indications

- Moderate unicompartmental gonarthrosis.
- Post-surgery cartilage or meniscus treatments.

### Contraindications

- Currently no known



ORTHO-A

## Orthosis for gonarthrosis



www.orthoservice.com



**Headquarter: ORTHOSERVICE AG**  
 Via Milano 7 - CH-6830 Chiasso (TI) - Switzerland  
 Tel. 0041 (0) 91 822 00 88 - Fax 0041 (0) 91 822 00 89  
 info@orthoservice.com - www.orthoservice.com  
**Niederlassung Deutschland: Orthoservice Deutschland GmbH**  
 Flugstraße 8 - D-76532 Baden-Baden - Deutschland  
 Tel. 0049 (0) 7221 991 39 11 - Fax 0049 (0) 7221 991 39 13  
 info@orthoservice.de - www.orthoservice.de  
**Orthoservice Polska sp. z o.o.**  
 ul. Warszawska 416a - 42-209 Częstochowa - Polska  
 Tel. +48 (0) 34 340 13 10  
 info@orthoservice.pl - www.orthoservice.pl  
**Sede italiana: RO+TEN s.r.l.**  
**Sede legale:** Via Marco De Marchi, 7 - I-20121 Milano (MI) - Italia  
**Sede operativa e amministrativa:**  
 Via Comasina, 111 - I-20843 Verano Brianza (MB) - Italia  
 Tel. 0039 039 601 40 94 - Fax 0039 039 601 42 34  
 info@roplusten.com - www.roplusten.com  
 Società soggetta a Direzione e Coordinamento (art. 2497bis CC):  
 Orthoservice AG (CH) - 6830 Chiasso (TI) - Switzerland



**ORTHOSERVICE**  
**RO+TEN**  
 Take care feel better



**ORTHOSERVICE**  
**RO+TEN**  
 Take care feel better



**LIGHT STRUCTURE MADE OF FOAM AND ALUMINUM ALLOY**

Two super-strong yet lightweight materials working together to facilitate joint realignment and pain reduction.

**WEDGES TO LIMIT EXTENSION AND FLEXION**

It limits flexion and extension to the degree prescribed by the orthopaedic surgeon.

**FOAM PAD**

Soft against the skin, the hypoallergenic pad remains securely in place. Easy to remove and wash in lukewarm water.

**FOUR NUMBERED PADDED STRAPS**

To facilitate closing.

**DOUBLE REGULATION JOINT OF MEDIAL AND LATERAL LOAD**

It enables the brace's condyle pad to be moved medially or laterally to create the right amount of "force" on the joint needed to correct its position.

NEW

NEW

**LOCKING BUCKLES WITH SAFETY BUTTON**

Never lose the correctly adjusted length of the straps and always have just the right amount of tension with minimal effort.



A painful knee is crying out for help. We provide an immediate solution.