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 **PLEASE READ THESE INSTRUCTIONS CAREFULLY AND KEEP THEM**

Ankle support in Aer-OS fabric

DECLARATION OF CONFORMITY

As the manufacturer, ORTHOSERVICE AG declares, under its sole responsibility, that this is a class I medical device that has been manufactured and assessed, according to the EU Regulation 2017/745 (MDR). These instructions have been drawn up in application of the aforementioned Regulation. They are intended to ensure an adequate and safe use of the medical device.

TRADEMARKS OF MATERIALS

Velcro® is a registered trademark of Velcro Industries B.V.

SAFETY PRECAUTIONS

We recommend that the pressure exerted by the device does not act on parts of the body with wounds, swelling, or welts.

It is advisable to not over tighten the device so as to avoid generating areas of excessive local pressure or the compression of underlying nerves and/or blood vessels. Furthermore, we recommend wearing a garment to avoid direct contact with the skin. If in doubt on how to apply the device, contact a doctor, physiotherapist, or an orthopedic technician. Read the product composition on the internal label carefully. We recommend to not wear the device in the proximity of free flames. Do not apply in direct contact with open wounds.

WARNINGS

It is advisable that the device, designed for the specific indications as set out below, be prescribed by a doctor or a physiotherapist and applied by an orthopedic technician, according to the specific needs of the patient. To ensure effectiveness, tolerability, and proper functioning, the application must be carried out with the utmost care. Never alter the adjustment made by the doctor/physiotherapist/ or orthopedic technician. The responsibilities of the manufacturer shall lapse in case of inappropriate use or adaptation. The orthosis is made and to be used by only one patient.

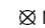
Should the orthosis be inappropriately used, the manufacturer declines all responsibility, as provided by the regulation for medical devices. In hypersensitive individuals, direct contact with the skin may cause redness and irritation. In case of pain, swelling, welts, or any other adverse reaction, contact your doctor immediately, and in case of a particularly severe adverse event, report the incident to the manufacturer and the competent authority of your country. The effectiveness of the orthopedic product is only guaranteed when all the components are properly in use.

SELECTION/SIZE

Code	REF.A1137			
Size	S	M	L	XL
Ankle circumf. cm	15/20	20/25	25/30	30/36
Colour	black			
fits right and left				

MAINTENANCE

 Do not bleach  No chemical cleaning

 Do not iron  Do not tumble-dry

 Washing instructions: hand wash at max. 30° C with neutral soap.

Dry away from heat sources.

Do not dispose of the product or any of its components into the environment.

INDICATIONS

- 1st degree tibiotarsal sprains
- Peroneal or instep tendinopathy

CONTRAINDICATIONS

Currently no known

FEATURES AND MATERIALS

- Made from breathable Aer-OS fabric
- Open heel

PUTTING ON THE APPLIANCE

1 Pull the bandage over the foot so that the orange label faces upwards and the orange profile is on the back (fig. A).

2 Bring the bandage into the correct position; ensure that the heel is positioned precisely in the heel cut-out (fig. B).

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Velcro® è un marchio di fabbrica registrato di Velcro Industries B.V.