# object

REF. 8237

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## ENGLISH

Please read the instructions carefully and thoroughly

### DECLARATION OF CONFORMITY

As the manufacturer, ORTHOSERVICE AG declares with sole responsibility that this product is a Class I medical product and has been manufactured in accordance with the basic requirements of the directives 93/42/EEC and 2007/47/EEC. These operating instructions were created in accordance with the specification of the above-mentioned directives. They aim to ensure the safe and correct use of the medical product.

### TRADEMARKS OF MATERIALS

Breath-o-prene® is a registered trademark of Nexus Foams Ltd.

### SELECTION/SIZE

REF. 8237				
Size	S	M	L	XL
Ankle circumf. cm	15/20	20/25	25/30	30/36
Colour: beige				
fits right and left				

### MAINTENANCE

- Do not bleach
- No chemical cleaning
- Do not iron
- Do not tumble-dry
- Washing instructions: hand wash at max. 30° C with neutral soap (a sponge is recommended); do not dry in the vicinity of heat sources

Do not dispose of the product or any of its components into the environment.

### PRECAUTIONS

It is advised that the pressure exerted by the product does not act on parts of the body suffering from wounds, swelling or lumps. The product should not be tightened too firmly so as to avoid the development of pressure points or compression of the nerves or blood vessels underneath. It is advisable to wear the product over a garment, avoiding direct contact with the skin. If in doubt about how to apply, contact an orthopaedic technician. The device should not be worn in the vicinity of open flames.

### WARNINGS

The product may only be used in the event of medical indication or when prescribed by a physician and must be applied by an orthopaedic technician, the competent authority figure for the application as well as for information on the safe use, according to individual needs. To ensure the effectiveness, tolerability, and correct operation, it is necessary that the device is put on with the utmost care. Any changes to the structure or adjustment of the device must be prescribed by a physician and performed by an orthopaedic technician. Under no circumstances should the adjustment made by the physician/orthopaedic technician be altered. Use of the product is recommended for a single patient. With hypersensitive patients, direct contact with skin may cause redness or irritation. In the case of onset of pain, swelling, lumps or any unusual reaction, immediately contact your physician. Individuals who are suffering from dermatitis or allergies to products containing neoprene should not use this product.

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- Double-lined orthopaedic neoprene
- Open heel
- Washable

### INDICATIONS

- Grade 1 ankle sprains
- Tendinopathy of the instep and peroneal tendons

### CONTRAINDICATIONS

Currently no known

### PUTTING ON THE APPLIANCE

- 1 Slide on the ankle support with the writing facing upwards and the seam at the front (A).
- 2 Position the ankle support so that the heel fits perfectly in the opening provided (B).

### WARRANTY

This warranty is governed by the statutory provisions of the country in which the product was purchased. Where relevant that country's regulations concerning warranties and the relationship between the seller and buyer shall apply. If you believe that the terms of the warranty may be applicable, first contact the retailer where the product was purchased. Orthoservice will repair or replace all or part of the device and its accessories if defects in materials and workmanship arise during the period of six months from the date of purchase.

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Via Milano 7 · CH-6830 Chiasso (TI) · Switzerland

Tel. 0041 91 8220088 · Fax 0041 91 8220089

info@orthoservice.com · www.orthoservice.com

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Tel. 0049 (0)7221 9913911 · Fax 0049 (0)7221 9913913

info@orthoservice.de · www.orthoservice.de

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