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## PLEASE READ THESE INSTRUCTIONS CAREFULLY AND KEEP THEM

REF. 530 - REF. 520: Walker with (two) inflatable air padding  
REF. 505: Diabetic Set

### DECLARATION OF CONFORMITY

As the ORTHOSERVICE AG declares, under its sole responsibility, that this is a class I medical device that has been manufactured and assessed, according to the EU Regulation 2017/745 (MDR). These instructions have been drawn up in application of the aforementioned Regulation. They are intended to ensure an adequate and safe use of the medical device.

### TRADEMARKS OF MATERIALS

Velcro® is a registered trademark of Velcro Industries B.V.

### SAFETY PRECAUTIONS

We recommend that the pressure exerted by the device does not act on parts of the body with wounds, swelling, or welts. It is advisable to not over tighten the device so as to avoid generating areas of excessive local pressure or the compression of underlying nerves and/or blood vessels. Furthermore, we recommend wearing a garment to avoid direct contact with the skin. If in doubt on how to apply the device, contact a doctor, physiotherapist, or an orthopedic technician. Read the product composition on the internal label carefully.

We recommend not to wear the device in the proximity of free flames or strong electromagnetic fields.

Do not apply in direct contact with open wounds.

### WARNINGS

It is advisable that the device, designed for the specific indications as set out below, be prescribed by a doctor or a physiotherapist and applied by an orthopedic technician, according to the specific needs of the patient. To ensure effectiveness, tolerability, and proper functioning, the application must be carried out with the utmost care. Never alter the adjustment made by the doctor/physiotherapist/ or orthopedic technician. The responsibilities of the manufacturer shall lapse in case of inappropriate use or adaptation. The orthosis is made and to be used by only one patient. Should the orthosis be inappropriately used, the manufacturer declines all responsibility, as provided by the regulation for medical devices. In hypersensitive individuals, direct contact with the skin may cause redness and irritation. In case of pain, swelling, welts, or any other adverse reaction, contact your doctor immediately, and in case of a particularly severe adverse event, report the incident to the manufacturer and the competent authority of your country. The effectiveness of the orthopedic product is only guaranteed when all the components are properly in use.

### SELECTION/SIZE

Item	REF.520 - REF.530 - REF.505		
Size	S	M	L
Shoe size	35/39	39/44	>44
Colour	grey		

fits right and left

### MAINTENANCE

- Do not bleach
- Do not iron
- One patient only
- Wash instructions:
  - Padding: Wash by hand in lukewarm water (max. 30°C) with neutral soap; rinse thoroughly. Dry away from heat sources
  - Rigid parts: Clean with a sponge soaked in lukewarm water (max. 30°C) and neutral soap. Dry with a cloth

Do not dispose of the product or any of its components into the environment.

### INDICATIONS

- Particularly suited for use with patients with sensitivities, for example, diabetes mellitus and the associated diabetic foot syndrome
- Ulcer on the metatarsus, malleolus, hallux or heel
- Charcot foot

### CONTRAINDICATIONS

REF.505: No known

REF.520-530: The application of this brace is advised only when it is documented that the fracture is stable and there are acceptable limits of angular and rotational misalignments

### CHARACTERISTICS AND MATERIALS

REF.505

- Adaptable foam inserts for partial pressure relief of the foot sole
- Special stockings for diabetes mellitus and the associated diabetic foot syndrome
- The diabetic set is designed for use with the Airstep Walker and Airstep tight Walker products

REF.520

- Adaptable foam inserts for partial pressure relief from the sole
- Special stockings for diabetes mellitus and the associated diabetic foot syndrome
- Front reinforcement shell for stable hold of the foot
- Optimum wearing comfort thanks to two individual inflatable air pads in the inner shoe
- Basic body made of durable, light plastic
- Comfortable wear, thanks to the foam padding
- Correct fixing thanks to the circular Velcro® fastening
- Optimized tilting sole
- Optional padding for further adaptation to the leg

REF.530

- Adaptable foam inserts for partial pressure relief from the sole
- Special stockings for diabetes mellitus and the associated diabetic foot syndrome
- Reinforcing shells on the front and back, for increased stability
- Inflatable air pads for optimum fit to the limb and improved stability
- Structure in lightweight, sturdy plastic