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PLEASE READ THESE INSTRUCTIONS CAREFULLY AND KEEP THEM

Wedges for Walker

DECLARATION OF CONFORMITY

As the manufacturer, ORTHOSERVICE AG declares, under its sole responsibility, that this is a class I medical device that has been manufactured and assessed, according to the EU Regulation 2017/745 (MDR). These instructions have been drawn up in application of the aforementioned Regulation. They are intended to ensure an adequate and safe use of the medical device.

MATERIALS

Eva; lactice free

SAFETY PRECAUTIONS

We recommend that the pressure exerted by the device does not act on parts of the body with wounds, swelling, or welts. It is advisable to not over tighten the device so as to avoid generating areas of excessive local pressure or the compression of underlying nerves and/or blood vessels. Furthermore, we recommend wearing a garment to avoid direct contact with the skin. If in doubt on how to apply the device, contact a doctor, physiotherapist, or an orthopedic technician. Please read the composition of the product on the inside label carefully; do NOT use the product if the patient has known intolerances or allergies to one or more components in the product. We recommend to not wear the device in the proximity of free flames. Do not apply in direct contact with open wounds.

WARNINGS

It is advisable that the device, designed for the specific indications as set out below, be prescribed by a doctor or a physiotherapist and applied by an orthopedic technician, according to the specific needs of the patient. To ensure effectiveness, tolerability, and proper functioning, the application must be carried out with the utmost care. Never alter the adjustment made by the doctor/physiotherapist/ or orthopedic technician. The responsibilities of the manufacturer shall lapse in case of inappropriate use or adaptation. The orthosis is made and to be used by only one patient. Should the orthosis be inappropriately used, the manufacturer declines all responsibility, as provided by the regulation for medical devices. In hypersensitive individuals, direct contact with the skin may cause redness and irritation. In case of pain, swelling, welts, or any other adverse reaction, contact your doctor immediately, and in case of a particularly severe adverse event, report the incident to the manufacturer and the competent authority of your country. The effectiveness of the orthopedic product is only guaranteed when all the components are properly in use.

DESCRIPTION

Wedges for Walker are to be used exclusively for individual adaptation in the orthosis Airstep Walker. They are useful only for the gradual adjustment of the heel area in the equine position.

MAINTENANCE

- ⚠ Do not bleach ✖ No chemical cleaning
- ⚠ Do not iron ✖ Do not tumble-dry
- 🧼 Washing instructions: wash by hand in lukewarm water (max. 30°C) with neutral soap. Dry away from heat sources.

Do not dispose of the product or any of its components into the environment.

INDICATIONS

- Heel wedges are designed for postoperative treatment of Achilles tendon reconstruction. Releases can be used immediately after treatment.

CONTRAINDICATIONS

Currently no known.

CHARACTERISTICS AND MATERIALS

- The set contains 5 wedges in different sizes.

CHARACTERISTICS

The set contains 5 heel wedges in different size. Regardless of foot shape and size, the 3 wedges can be positioned one above the other and be fixed in the heel bar at the heel. For the small orthoses the 3 smallest wedges are usually used, for large orthoses the 3 largest and for the medium ones 3 medium sized ones.

PUTTING ON THE APPLIANCE

- 1 Remove the adhesive film from the lower part of the larger wedge. Attach the wedge in the heel area to the brace plant. (figure A).
- 2 Remove the adhesive film from the other wedge. Fix the other two parts on the lower part of the wedge. (fig. B/C)
- 3 Use as many heights as you need for the right heel angle (fig. D)
1 wedge = 10°, 2 wedges = 16°, 3 wedges = 22°, 4 wedges = 28°, 5 wedges = 30°.

If is necessary, remove the wedges during the healing process.

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