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 PLEASE READ THESE INSTRUCTIONS CAREFULLY AND KEEP THEM

REF.303 Viscoelastic heel support
REF.304INT Viscoelastic heel support with heel bone hole
REF.304SS Viscoelastic heel support with heel bone hole
REF.311 Viscoelastic metatarsal padding
REF.312 Viscoelastic longitudinal arch support

DECLARATION OF CONFORMITY

ORTHOSERVICE AG declares, under its sole responsibility, that this is a class I medical device that has been manufactured and assessed, according to the EU Regulation 2017/745 (MDR). These instructions have been drawn up in application of the aforementioned Regulation. They are intended to ensure an adequate and safe use of the medical device.

SAFETY PRECAUTIONS

We recommend that the pressure exerted by the device does not act on parts of the body with wounds, swelling, or welts. It is advisable to not over tighten the device so as to avoid generating areas of excessive local pressure or the compression of underlying nerves and/or blood vessels.

Furthermore, we recommend wearing a garment to avoid direct contact with the skin. If in doubt on how to apply the device, contact a doctor, physiotherapist, or an orthopedic technician. Please read the composition of the product on the inside label carefully; do NOT use the product if the patient has known intolerances or allergies to one or more components in the product. We recommend to not wear the device in the proximity of free flames. Do not apply in direct contact with open wounds.

WARNINGS

It is advisable that the device, designed for the specific indications as set out below, be prescribed by a doctor or a physiotherapist and applied by an orthopedic technician, according to the specific needs of the patient.

To ensure effectiveness, tolerability, and proper functioning, the application must be carried out with the utmost care. Never alter the adjustment made by the doctor/physiotherapist/or orthopedic technician. The responsibilities of the manufacturer shall lapse in case of inappropriate use or adaptation. The orthosis is made and to be used by only one patient. Should the orthosis be inappropriately used, the manufacturer declines all responsibility, as provided by the regulation for medical devices. In hypersensitive individuals, direct contact with the skin may cause redness and irritation. In case of pain, swelling, welts, or any other adverse reaction, contact your doctor immediately, and in case of a particularly severe adverse event, report the incident to the manufacturer and the competent authority of your country. The effectiveness of the orthopedic product is only guaranteed when all the components are properly in use.

SELECTION/SIZE

Code	REF.303 • REF.304INT • REF.304SS • REF.311 • REF.312
Size	S M L
Shoe size	35/39 40/43 44/48
Color	blu

Supplied in pairs

MAINTENANCE

- Washing instructions: hand wash in tepid 30°C water with mild soap. Leave to dry away from direct heat.
- Dispose of this product responsibly

INDICATIONS

- REF.303**
 - For mild heel pathologies.
- REF.304INT - REF. 304SS**
 - For mild heel bone pathologies.
- REF.311**
 - For mild metatarsal pathologies.
- REF.312**
 - For mild longitudinal arch pathologies.

CONTRAINDICATIONS

Currently no known.

FEATURES AND MATERIALS

- Structure in visco-elastic material
- Upper lining in soft felt
- Washable
- REF.304INT**
- Central hole for the placement of the calcaneal spine
- Biomechanical and precise shock absorption
- REF.304SS**
- Directional hole for the placement of the calcaneal spine
- Biomechanical and precise shock absorption
- REF. 311** Longitudinal grooves for the placement of the metatarsals
- REF. 312** Central relief to support the longitudinal arch of the foot
- REF. 312** Can be used either individually or combined with an insole

INSTRUCTIONS FOR USE

- Remove the protective film.
- If necessary, trim with scissors.
- Place in the shoe with the felt part facing upwards.

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