duraband

REF.PB6001 – REF.PB6006
REF.PB6008 – REF.PB6013

Elastisches Band aus Natur-Latex 5 m - 30 m
Elastic natural latex band 5 m - 30 m
Bande élastique en latex naturel 5 m - 30 m
Эластичная лента из натурального латекса длиной 5 - 30 метров
Banda elastica in lattice naturale 5 m - 30 m

DEUTSCH

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ENGLISH

Please read the instructions carefully and thoroughly

DECLARATION OF CONFORMITY

As the manufacturer, ORTHOSERVICE AG states under its sole responsibility that this product is a Class I medical device that has been manufactured in accordance with the requirements of Directives 93/42/EEC and 2007/47/EC. These instructions have been written in conformity with these directives, and their aim is to ensure the safe and proper use of this medical device.

SELECTION/SIZE

PB 6001	yellow	light	0,15 mm x 5 m
PB 6002	orange	moderate	0,20 mm x 5 m
PB 6003	green	medium	0,25 mm x 5 m
PB 6004	pink	strong	0,30 mm x 5 m
PB 6005	purple	super strong	0,40 mm x 5 m
PB 6006	silver	extra strong	0,45 mm x 5 m
PB 6008	yellow	light	0,15 mm x 30 m
PB 6009	orange	moderate	0,20 mm x 30 m
PB 6010	green	medium	0,25 mm x 30 m
PB 6011	pink	strong	0,30 mm x 30 m
PB 6012	purple	super strong	0,40 mm x 30 m
PB 6013	silver	extra strong	0,45 mm x 30 m

Colour: 6 different resistances (colours)
Option: Pair of handles for Duraband (REF. PB6007)

Elongation in %	Resistance in kg					
	Light	Moderate	Medium	Strong	Super strong	Extra strong
66%	1,02	1,58	1,94	2,83	3,39	5,46
100%	1,12	2,04	2,27	3,39	4,08	6,81
133%	1,35	2,50	2,73	4,08	4,54	7,88
166%	1,58	2,96	3,19	4,64	5,10	9,08
200%	1,81	3,39	3,62	5,46	5,89	10,43

RESISTANCE VALUES

The table with the resistance values is useful to have a reference point prior to starting the training. It helps to calculate the resistance required to correctly perform the exercise, in relation to the stretch percentage of the band. For example, if you stretch a band with an initial length of 30 cm to 60 cm, the elongation/stretch percentage is 100%, and the force required for this elongation is 1.12 kg for the yellow band (light) and 6.81 kg for the grey band (extra strong). Tip: The band can be converted into a loop with a simple knot. The value of a 30 cm long loop doubles to the value indicated in the table. After the exercise, the knot can be simply undone.

MAINTENANCE

Store the band at room temperature in a box, in a dark place and protected from sunlight. Keep the band away from sharp objects, be careful with long fingernails, and remove any rings prior to use. Rinse the band under running water and then leave to dry naturally. After the band has dried completely, dust it with talcum powder.

PRECAUTIONS

Prior to use, check the band for cuts, small cracks or holes, as these might lead to a sudden tearing of the band. If the band is damaged, dispose of the product and replace it with a new one.

WARNINGS

The Duraband for cardio/strength and rehabilitation exercises may cause serious injury if used incorrectly. We recommend to use the band only on recommendation of a physician or a professional person, and to follow the illustrated instructions. The Duraband must not be used for persons under the age of 8 and, in any case, only under the supervision of an adult. This product contains natural latex and can cause allergies. Do not swallow.

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EN – The descriptions and images in this document are for example and commercial purposes only. Orthoservice reserves the right to modify them according to its needs.

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ENGLISH

1 PAIR HANDLES FOR DURABAND (REF. PB6007)

Using the handles, which have a special fixing system, the exercises can be performed in a more efficient and safer manner. Simply guide the Duraband rubber band through the first and second fixing opening and then pull tight.

CHARACTERISTICS AND MATERIALS

- Elastic natural latex band for cardio training /strength training and rehabilitation exercises

INDICATIONS

- Training after surgical interventions

CONTRAINDICATIONS

Currently no known

WARRANTY

This warranty is governed by the statutory provisions of the country in which the product was purchased. Where relevant that country's regulations concerning warranties and the relationship between the seller and buyer shall apply. If you believe that the terms of the warranty may be applicable, first contact the retailer where the product was purchased. Orthoservice will repair or replace all or part of the device and its accessories if defects in materials and workmanship arise during the period of six months from the date of purchase.

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