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REF.KS-ZG-001

Canadian crutches

DESIGNED AND DEVELOPED IN SWITZERLAND

ORTHOSERVICE RO+TEN
Take care feel better

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EN – The descriptions and images in this document are for illustrative and commercial purposes only. Orthoservice reserves the right to make modifications without prior notice based on their needs.

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OSUK Ltd
Units 47&50 North East BIC
Wearfield
Sunderland Enterprise Park East
Sunderland SR5 2TA
Tel +44 (0) 191 51 66 220
info@orthoservice.uk · www.orthoservice.uk

RO+TEN S.r.l.
Sede legale: Via Marco De Marchi 7 · I-20121 Milano (MI) · Italia
Sede operativa e amministrativa:
Via Comasina 111 · I-20843 Verano Brianza (MB) · Italia
Tel. 0039 0362 18 51 200
info@roplusten.com · www.roplusten.com

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 PLEASE READ THESE INSTRUCTIONS CAREFULLY AND KEEP THEM

Canadian crutches

DECLARATION OF CONFORMITY

As the manufacturer, ORTHOSERVICE AG declares, under its sole responsibility, that this is a class I medical device that has been manufactured and assessed, according to the EU Regulation 2017/745 (MDR). These instructions have been drawn up in application of the aforementioned Regulation. They are intended to ensure an adequate and safe use of the medcal device.

SAFETY PRECAUTIONS

Do not use the product if there are wounds, swellings or swellings in the area of contact with the product. If in doubt on how to use it, consult a doctor, physiotherapist or orthopaedic technician. It is advisable not to wear the device near open flame devices or strong electromagnetic fields. Do not use in direct contact with open wounds. To avoid falls, be careful of wet and slippery surfaces. Do not use a damaged device. Avoid soaking in water, as the water would enter through the holes and stagnate. Maximum user weight: 80 kg. Please read the composition of the product carefully; do NOT use the product if the patient has known intolerances or allergies to one or more components in the product.

WARNINGS

The product is designed for the specific indications below and should be prescribed by a doctor or physiotherapist and adjusted by an orthopaedic technician, in accordance with individual requirements. To ensure effectiveness, tolerability and proper functioning, the adjustment must be carried out with the utmost care. The adjustment carried out by the doctor/ physiotherapist/orthopaedic technician must not be altered under any circumstances. The manufacturer avoids liability in the event of inappropriate use or adjustment. The orthosis is intended for use by one patient only; otherwise the manufacturer accepts no liability in accordance with the regulations for medical devices. In hypersensitive individuals, direct skin contact may cause redness or irritation. In the event of pain, swelling, tumefaction or any other abnormal reaction, seek medical advice immediately and, if particularly serious, report the fact to the manufacturer and the competent authority in your country. In some people, prolonged use can cause problems with the arm joints; if you feel pain in your hand, wrist, elbow or shoulder, it is advisable to stop using the aid and consult a doctor.

SELECTION/SIZE

Code	REF.KS-ZG-001
Size	universal
Color	blue

MAINTENANCE

CARE GUIDELINES

- Clean regularly (especially the handle) with a sponge soaked in lukewarm water (30° C) and mild soap. Dry with a cloth.
- Do not use alcohol or solvents
- Should water enter through the holes, disassemble the tubes and the tip and allow to dry.

When worn, replace the tip with original spare parts.

Do not dispose of the product or any of its components into the environment.

INDICATIONS

- Walking aid to help relieve the lower limb of body weight

CONTRAINDICATIONS

Currently no known

FEATURES AND MATERIALS

- Lightweight cushioned aluminium alloy crutches
- Adjustable at the touch of a button
- Ergonomic polypropylene handle
- Rubber tip

PUTTING ON THE APPLIANCE

- Press the button and slide the lower part to the desired height.
- Make sure that the button is firmly in place.
- The height should be defined according to the patient's height, comfort and subjective manoeuvrability. The elbow should remain slightly flexed during use.

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