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**ORTHO14-810**

REF.OFT105I - REF.OFT105P

## Bivalve cervical collar with tracheal hole (pediatric)

 **DESIGNED AND DEVELOPED IN SWITZERLAND**

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EN – The descriptions and images in this document are for illustrative and commercial purposes only. Orthoservice reserves the right to make modifications without prior notice based on their needs.

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**OSUK Ltd**  
Units 47&50 North East BIC  
Wearfield  
Sunderland Enterprise Park East  
Sunderland SR5 2TA  
Tel +44 (0) 191 51 66 220  
info@orthoservice.uk · www.orthoservice.uk

**RO+TEN S.r.l.**  
Sede legale: Via Marco De Marchi 7 · I-20121 Milano (MI) · Italia  
Sede operativa e amministrativa:  
Via Comasina 111 · I-20843 Verano Brianza (MB) · Italia  
Tel. 0039 0362 18 51 200  
info@roplusten.com · www.roplusten.com

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PLEASE READ THESE INSTRUCTIONS CAREFULLY AND KEEP THEM

**Bivalve cervical collar with tracheal hole (pediatric)**

**DECLARATION OF CONFORMITY**

As the manufacturer, ORTHOSERVICE AG declares, under its sole responsibility, that this is a class I medical device that has been manufactured and assessed, according to the EU Regulation 2017/745 (MDR). These instructions have been drawn up in application of the aforementioned Regulation. They are intended to ensure an adequate and safe use of the medical device.

**TRADEMARKS OF MATERIALS**

Velcro® is a registered trademark of Velcro Industries B.V.

**SAFETY PRECAUTIONS**

We recommend that the pressure exerted by the device does not act on parts of the body with wounds, swelling, or welts. It is advisable to not over tighten the device so as to avoid generating areas of excessive local pressure or the compression of underlying nerves and/or blood vessels. If in doubt on how to apply the device, contact a doctor, physiotherapist, or an orthopedic technician. Please read the composition of the product on the inside label carefully; do NOT use the product if the patient has known intolerances or allergies to one or more components in the product. We recommend to not wear the device in the proximity of free flames. Do not apply in direct contact with open wounds.

**WARNINGS**

It is advisable that the device, designed for the specific indications as set out below, be prescribed by a doctor or a physiotherapist and applied by an orthopedic technician, according to the specific needs of the patient. To ensure effectiveness, tolerability, and proper functioning, the application must be carried out with the utmost care. Never alter the adjustment made by the doctor/physiotherapist/ or orthopedic technician. The responsibilities of the manufacturer shall lapse in case of inappropriate use or adaptation. The orthosis is made and to be used by only one patient. Should the orthosis be inappropriately used, the manufacturer declines all responsibility, as provided by the regulation for medical devices. In hypersensitive individuals, direct contact with the skin may cause redness and irritation. In case of pain, swelling, welts, or any other adverse reaction, contact your doctor immediately, and in case of a particularly severe adverse event, report the incident to the manufacturer and the competent authority of your country. The effectiveness of the orthopedic product is only guaranteed when all the components are properly in use.

**SELECTION/SIZE**

Code	REF.OFT105I	REF.OFT105P
Size	kids 0-3 years old	kids 4-9 years old
Color	beige	

**MAINTENANCE**

- Do not bleach  No chemical cleaning
- Do not iron  Do not tumble-dry
- Washing instructions: hand wash at max. 30°C with neutral soap. Dry away from heat sources.

Do not dispose of the product or any of its components into the environment.

**INDICATIONS**

- Medium severity to severe distortion of the cervical spine (whiplash)
- Multiple discopathies of the cervical spine
- Cervical spine traumas
- Pathological instability as a result of cervical spine metastases
- Support of post-operative progression in spinal surgery

**CONTRAINDICATIONS**

Currently no known

**CHARACTERISTICS AND MATERIALS**

- Non-toxic PLASTAZOTE® structure
- Consists of two enveloping shells with Velcro® closure
- Central front and back rigid support plates
- Radiolucent
- Shaped for perfect fit at chin, neck and shoulders
- Tracheal hole

**PUTTING ON THE APPLIANCE**

- 1 Position the front shell so that the chin is resting in the proper position
- 2 Position the rear shell and place the side loops over one another, in doing so, those on the rear shell should be inside and those on the front shell should be outside
- 3 Pull both loops evenly at both sides and secure well using the Velcro® closures

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