 LŐDZU, UZMANÍGI IZLASIET UN SAGLABÁJIEI INSTRUKCIJAS

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REF.OFT315

Clavicle splint (pediatric)

DESIGNED AND DEVELOPED IN SWITZERLAND

ORTHOSERVICE
RO+TEN
Take care feel better

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EN – The descriptions and images in this document are for illustrative and commercial purposes only. Orthoservice reserves the right to make modifications without prior notice based on their needs.

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OSUK Ltd
Units 47&50 North East BIC
Wearfield
Sunderland Enterprise Park East
Sunderland SR5 2TA
Tel +44 (0) 191 51 66 220
info@orthoservice.uk · www.orthoservice.uk

RO+TEN S.r.l.
Sede legale: Via Marco De Marchi 7 · I-20121 Milano (MI) · Italia
Sede operativa e amministrativa:
Via Comasina 111 · I-20843 Verano Brianza (MB) · Italia
Tel. 0039 0362 18 51 200
info@roplusten.com · www.roplusten.com

SINGLE PATIENT
MULTIPLE USE

LATEX
FREE

RADIO
LUCENT

ORTHOSERVICE
RO+TEN
Take care feel better

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 PLEASE READ THESE INSTRUCTIONS CAREFULLY AND KEEP THEM

Clavicle splint (pediatric)

DECLARATION OF CONFORMITY

As the manufacturer, ORTHOSERVICE AG declares, under its sole responsibility, that this is a class I medical device that has been manufactured and assessed, according to the EU Regulation 2017/745 (MDR). These instructions have been drawn up in application of the aforementioned Regulation. They are intended to ensure an adequate and safe use of the medcal device.

TRADEMARKS OF MATERIALS

Velcro® is a registered trademark of Velcro Industries B.V.

SAFETY PRECAUTIONS

We recommend that the pressure exerted by the device does not act on parts of the body with wounds, swelling, or welts. It is advisable to not over tighten the device so as to avoid generating areas of excessive local pressure or the compression of underlying nerves and/or blood vessels. Furthermore, we recommend wearing a garment to avoid direct contact with the skin. If in doubt on how to apply the device, contact a doctor, physiotherapist, or an orthopedic technician. Please read the composition of the product on the inside label carefully; do NOT use the product if the patient has known intolerances or allergies to one or more components in the product. We recommend to not wear the device in the proximity of free flames or strong electromagneticfields. Do not apply in direct contact with open wounds.

WARNINGS

It is advisable that the device, designed for the specific indications as set out below, be prescribed by a doctor or a physiotherapist and applied by an orthopedic technician, according to the specific needs of the patient.

To ensure effectiveness, tolerability, and proper functioning, the application must be carried out with the utmost care. Never alter the adjustment made by the doctor/physiotherapist/ or orthopedic technician. The responsibilities of the manufacturer shall lapse in case of inappropriate use or adaptation. The orthosis is made and to be used by only one patient. Should the orthosis be inappropriately used, the manufacturer declines all responsibility, as provided by the regulation for medical devices.In hypersensitive individuals, direct contact with the skin may cause redness and irritation. In case of pain, swelling, welts, or any other adverse reaction, contact your doctor immediately, and in case of a particularly severe adverse event, report the incident to the manufacturer and the competent authority of your country. The effectiveness of the orthopedic product is only guaranteed when all the components are properly in use.

SELECTION/SIZE

Code	REF.OFT315
Size	universal
Colour	white
fits right and left	

MAINTENANCE

- Do not bleach

No chemical cleaning
- Do not iron

Do not tumble-dry
- Washing instructions: hand wash at max. 30°C with neutral soap.

Dry away from heat sources.

Do not dispose of the product or any of its components into the environment.

INDICATIONS

- Clavicle fractures
- Subluxation of the acromionclavicular joint

CONTRAINDICATIONS

Currently no known

CHARACTERISTICS AND MATERIALS

- Made from gauze fabric with padding
- Applied similarly to the figure of eight bandage with Velcro® closure at the back

PUTTING ON THE APPLIANCE

- Place the padded parts on the shoulders, with the rear pad on the back and the straps hanging down
- Pass the straps under the armpits and through the loops. Tension the closing straps under the armpits until the desired fit is achieved.

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