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PLEASE READ THESE INSTRUCTIONS CAREFULLY AND KEEP THEM

Orthoself Line

DECLARATION OF CONFORMITY
As the manufacturer, ORTHOSERVICE AG declares, under its sole responsibility, that this is a class I medical device that has been manufactured and assessed, according to the EU Regulation 2017/745 (MDR). These instructions have been drawn up in application of the aforementioned Regulation. They are intended to ensure an adequate and safe use of the medical device.

SAFETY PRECAUTIONS
We recommend that the pressure exerted by the device does not act on parts of the body with wounds, swelling, or welts. It is advisable to not over tighten the device so as to avoid generating areas of excessive local pressure or the compression of underlying nerves and/or blood vessels. If in doubt on how to apply the device, contact a doctor, physiotherapist, or an orthopedic technician. Please read the composition of the product on the inside label carefully; do NOT use the product if the patient has known intolerances or allergies to one or more components in the product. We recommend to not wear the device in the proximity of free flames. Do not apply in direct contact with open wounds.

WARNINGS
It is advisable that the device, designed for the specific indications as set out below, be prescribed by a doctor or a physiotherapist and applied by an orthopedic technician, according to the specific needs of the patient. To ensure effectiveness, tolerability, and proper functioning, the application must be carried out with the utmost care. Never alter the adjustment made by the doctor/physiotherapist/ or orthopedic technician. The responsibilities of the manufacturer shall lapse in case of inappropriate use or adaptation. The orthosis is made and to be used by only one patient. Should the orthosis be inappropriately used, the manufacturer declines all responsibility, as provided by the regulation for medical devices. In hypersensitive individuals, direct contact with the skin may cause redness and irritation. In case of pain, swelling, welts, or any other adverse reaction, contact your doctor immediately, and in case of a particularly severe adverse event, report the incident to the manufacturer and the competent authority of your country. The effectiveness of the orthopedic product is only guaranteed when all the components are properly in use.

SELECTION/SIZE

Code	REF.19032 Epi-S32				
Size	XS	S	M	L	XL
Circumference cm*	20/23	23/27	27/31	31/35	35/40
Colour	black				

Fits right and left / *Forearm circumference

Code	REF.19019 Manu-S19				
Size	XS	S	M	L	XL
Circumference cm*	14,5/15,5	15,5/16,5	16,5/17,5	17,5/18,5	18,5/20
Colour	black				
Fits right and left / *Wrist circumference					

Code	REF.19033 Mio-S33				
Size	S	M	L	XL	XXL
Circumference cm*	45/48	48/52	52/56	56/61	61/67
Colour	black				
Fits right and left / *Thigh center circumference					

Code	REF.19060 Genu-S60					
Size	XS	S	M	L	XL	XXL
Circumference cm*	35/38	38/41	41/45	45/49	49/54	54/60
Colour	black					
Fits right and left / *Thigh circ. taken 15 cm above mid-patella cm						

Code	REF.19034 Mio-S34			
Size	S	M	L	XL
Circumference cm*	33/36	36/39	39/43	43/48
Colour	black			
Fits right and left / *Calf circumference				

Code	REF.19037 Malleo-S37				
Size	XS	S	M	L	XL
Circumference cm*	19/21	21/24	24/27	27/30	30/33
Colour	black				
Fits right and left / *Ankle circumference					

MAINTENANCE
☒ Do not bleach ☒ No chemical cleaning
☒ Do not iron ☒ Do not tumble-dry
☒ Washing instructions: hand wash at max. 30° C with neutral soap.
☒ Dry away from heat sources.
Do not dispose of the product or any of its components into the environment.

INDICATIONS
EPI-S 32
MANU-S 19

- Chronic pain from bursitis or tendinitis
- Aftercare for wrist sprains
- Tendinitis and tenosynovitis
- Inflammatory degenerative diseases of the wrist
- Muscular contractions and cramps of the thigh
- Aid to la reprise de la pratique sportive après une blessure

GENU-S 60

- Gonalgia from bruising
- Early stage arthritis
- Tendinitis

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MALLEO-S 37 : Bruising

• Tendinopathy of the instep and peronei

KONTRAINDICATIONS
Currently no known
FEATURES AND MATERIALS

- The stretchy tubular support is anatomically designed to fit the joint perfectly
- Breathable
- Medium to high compression
- Comfortable edges
- Composition: 70% polyamide, 30% elastane

NUR MANU-S 19 : Thumbs hole
ONLY GENU-S 60 : Nonslip silicone upper insert
ONLY MALLEO-S 37 : Special weave on the heel for a stronger, more durable design

PUTTING ON THE APPLIANCE
EPI-S 32

- Slide the tubular support with the orange label at the top onto the arm
- Position the support so that the label is centred on the back (above the elbow)

MANU-S 19

- Slide the tubular support onto the forearm with the orange label at the top
- Position the support so that the thumb fits into the hole provided.

MIO-S 33

- Slide the tubular support onto the thigh with the orange label at the top.
- Position the support so that the label is centred at the front.

GENU-S 60

- Slide the tubular support onto the thigh with the orange label at the top
- Position the support so that the label is centred at the front and the knee is in the middle of the support.

MIO-S 34

- Slide the tubular support onto the calf with the orange label at the top.
- Position the support so that the label is centred.

MALLEO-S 37

- Slide the tubular support with the orange label at the top.
- Position the support so that the labels centred at the front and the heel sits properly in the correct position.

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