## EPI-S32

REF.19032

Ellenbogen-Bandage

Elbow support

Coudière simple

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**PLEASE READ THESE INSTRUCTIONS CAREFULLY AND KEEP THEM**

**Orthoself Line**

**DECLARATION OF CONFORMITY**

As the manufacturer, ORTHOSERVICE AG declares, under its sole responsibility, that this is a class I medical device that has been manufactured and assessed, according to the EU Regulation 2017/745 (MDR). These instructions have been drawn up in application of the aforementioned Regulation. They are intended to ensure an adequate and safe use of the medical device.

**SAFETY PRECAUTIONS**

We recommend that the pressure exerted by the device does not act on parts of the body with wounds, swelling, or welts. It is advisable to not over tighten the device so as to avoid generating areas of excessive local pressure or the compression of underlying nerves and/or blood vessels. If in doubt on how to apply the device, contact a doctor, physiotherapist, or an orthopedic technician. Read the product composition on the internal label carefully. We recommend to not wear the device in the proximity of free flames. Do not apply in direct contact with open wounds.

**WARNINGS**

It is advisable that the device, designed for the specific indications as set out below, be prescribed by a doctor or a physiotherapist and applied by an orthopedic technician, according to the specific needs of the patient. To ensure effectiveness, tolerability, and proper functioning, the application must be carried out with the utmost care. Never alter the adjustment made by the doctor/physiotherapist/ or orthopedic technician.

The responsibilities of the manufacturer shall lapse in case of inappropriate use or adaptation.

The orthosis is made and to be used by only one patient. Should the orthosis be inappropriately used, the manufacturer declines all responsibility, as provided by the regulation for medical devices. In hypersensitive individuals, direct contact with the skin may cause redness and irritation.

In case of pain, swelling, welts, or any other adverse reaction, contact your doctor immediately, and in case of a particularly severe adverse event, report the incident to the manufacturer and the competent authority of your country. The effectiveness of the orthopedic product is only guaranteed when all the components are properly in use.

**SELECTION/SIZE**

<b>Code</b>	<b>REF.19032 Epi-S32</b>				
<b>Size</b>	<b>XS</b>	<b>S</b>	<b>M</b>	<b>L</b>	<b>XL</b>
<b>Circumference cm*</b>	20/23	23/27	27/31	31/35	35/40
<b>Colour</b>	black				
Fits right and left / *Forearm circumference					

<b>Code</b>	<b>REF.19019 Manu-S19</b>				
<b>Size</b>	<b>XS</b>	<b>S</b>	<b>M</b>	<b>L</b>	<b>XL</b>
<b>Circumference cm*</b>	14,5/15,5	15,5/16,5	16,5/17,5	17,5/18,5	18,5/20
<b>Colour</b>	black				
Fits right and left / *Wrist circumference					

<b>Code</b>	<b>REF.19033 Mio-S33</b>				
<b>Size</b>	<b>S</b>	<b>M</b>	<b>L</b>	<b>XL</b>	<b>XXL</b>
<b>Circumference cm*</b>	45/48	48/52	52/56	56/61	61/67
<b>Colour</b>	black				
Fits right and left / *High center circumference					

<b>Code</b>	<b>REF.19060 Genu-S60</b>					
<b>Size</b>	<b>XS</b>	<b>S</b>	<b>M</b>	<b>L</b>	<b>XL</b>	<b>XXL</b>
<b>Circumference cm*</b>	35/38	38/41	41/45	45/49	49/54	54/60
<b>Colour</b>	black					
Fits right and left / *Thigh circ. taken 15 cm above mid-patella cm						

<b>Code</b>	<b>REF.19034 Mio-S34</b>				
<b>Size</b>	<b>S</b>	<b>M</b>	<b>L</b>	<b>XL</b>	
<b>Circumference cm*</b>	33/36	36/39	39/43	43/48	
<b>Colour</b>	black				
Fits right and left / *Calf circumference					

<b>Code</b>	<b>REF.19037 Malleo-S37</b>				
<b>Size</b>	<b>XS</b>	<b>S</b>	<b>M</b>	<b>L</b>	<b>XL</b>
<b>Circumference cm*</b>	19/21	21/24	24/27	27/30	30/33
<b>Colour</b>	black				
Fits right and left / *Ankle circumference					

**MAINTENANCE**

Do not bleach No chemical cleaning No chemical cleaning

Do not iron Do not tumble-dry

Washing instructions: hand wash at max. 30° C with neutral soap.

Dry away from heat sources.

Do not dispose of the product or any of its components into the environment.

**INDICATIONS**

**EPI-S 32** • Chronic pain from bursitis or tendinitis

**MANU-S 19**

• Aftercare for wrist sprains

• Tendinitis and tenosynovitis

**MIO-S 33** • Inflammatory degenerative diseases of the wrist

• Muscular contractions and cramps of the thigh

• Aid to resuming sporting activity after trauma

**GENU-S 60**

• Gonalgia from bruising

• Early stage arthrosis

• Tendinitis

**MIO-S 34** • Muscular contractions and cramps of the calf

• Aid to resuming sporting activity after trauma

**MALLEO-S 37** • Bruising

• Tendinopathy of the instep and peronei

**KONTRAINDICATIONS**

Currently no known

**FEATURES AND MATERIALS**

• The stretchy tubular support is anatomically designed to fit the joint perfectly

• Breathable

• Medium to high compression

• Comfortable edges

• Composition: 70% polyamide, 30% elastane

**NUR MANU-S 19** Thumb hole

**ONLY GENU-S 60** Nonslip silicone upper insert

**ONLY MALLEO-S 37** Special weave on the heel for a stronger, more durable design

**PUTTING ON THE APPLIANCE**

**EPI-S 32** • Slide the tubular support with the orange label at the top onto the arm

• Position the support so that the label is centred on the back (above the elbow)