EPI-S32

REF:19032

Ellenbogen-Bandage
Elbow support
Coudière simple
Ортез - шина для кисти-запястья-локтевого сустава
Orteza lokcia
Elastiga bandaža elkoņa locītava
Kuynärpäätuki
Gomitiера semplice

MANU-S19

REF:19019

Handgelenk-Bandage
Wrist support
Orthèse de poignet simple
Ортез для запястья
Orteza nadgarstka
Ortoze plaukstas locītavas fiksācijai
Rannetuki
Polsiera semplice

MIO-S33

REF:19033

Oberschenkel-Bandage
Thigh support
Cuissard
Бандаж на бедро
Orteza uda
Elastiga augšstilba bandāža
Reisituki
Cosciale

GENU-S60

REF:19060

Knie-Bandage
Knee support
Genouillère simple
Ортез на коленный сустав
Orteza kolana
Elastiga bandaža ceļa locītavaī
Polivītuki
Ginocchiera semplice

MIO-S34

REF:19034

Wadenstütze
Calf support
Molletière
Бандаж на голень
Orteza roduždia
Elastīga arakšstilba bandāža
Pohjetuki
Supporto per polpaccio

MALLEO-S37

REF:19037

Knöchelbandage
Ankle support
Chevilière
Ортез для голеностопный сустав
Orteza stawi skokowego
Elastīga bandaža pēdas locītavaī
Nilkkatuki
Cavigliera

MADE WITH
MERZ
MACHINES

SEAMLESS

SINGLE PATIENT
MULTIPLE USE

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 PLEASE READ THESE INSTRUCTIONS CAREFULLY AND KEEP THEM

Orthoseff Line

DECLARATION OF CONFORMITY

As the manufacturer, ORTHOSERVICE AG declares, under its sole responsibility, that this is a class I medical device that has been manufactured and assessed, according to the EU Regulation 2017/745 (MDR). These instructions have been drawn up in application of the aforementioned Regulation. They are intended to ensure an adequate and safe use of the medical device.

SAFETY PRECAUTIONS

We recommend that the pressure exerted by the device does not act on parts of the body with wounds, swelling, or welts. It is advisable to not over tighten the device so as to avoid generating areas of excessive local pressure or the compression of underlying nerves and/or blood vessels. If in doubt on how to apply the device, contact a doctor, physiotherapist, or an orthopedic technician. Read the product composition on the internal label carefully. We recommend to not wear the device in the proximity of free flames. Do not apply in direct contact with open wounds.

WARNINGS

It is advisable that the device, designed for the specific indications as set out below, be prescribed by a doctor or a physiotherapist and applied by an orthopedic technician, according to the specific needs of the patient. To ensure effectiveness, tolerability, and proper functioning, the application must be carried out with the utmost care. Never alter the adjustment made by the doctor/physiotherapist/ or orthopedic technician.

The responsibilities of the manufacturer shall lapse in case of inappropriate use or adaptation.

The orthosis is made and to be used by only one patient. Should the orthosis be inappropriately used, the manufacturer declines all responsibility, as provided by the regulation for medical devices. In hypersensitive individuals, direct contact with the skin may cause redness and irritation.

In case of pain, swelling, welts, or any other adverse reaction, contact your doctor immediately, and in case of a particularly severe adverse event, report the incident to the manufacturer and the competent authority of your country. The effectiveness of the orthopedic product is only guaranteed when all the components are properly in use.

SELECTION/SIZE

Code	REF.19032 Epi-S32				
Size	XS	S	M	L	XL
Circumference cm*	20/23	23/27	27/31	31/35	35/40
Colour	black				
Fits right and left / *Forearm circumference					

Code	REF.19019 Manu-S19				
Size	XS	S	M	L	XL
Circumference cm*	14,5/15,5	15,5/16,5	16,5/17,5	17,5/18,5	18,5/20
Colour	black				
Fits right and left / *Wrist circumference					

Code	REF.19033 Mio-S33				
Size	S	M	L	XL	XXL
Circumference cm*	45/48	48/52	52/56	56/61	61/67
Colour	black				
Fits right and left / *High center circumference					

Code	REF.19060 Genu-S60					
Size	XS	S	M	L	XL	XXL
Circumference cm*	35/38	38/41	41/45	45/49	49/54	54/60
Colour	black					
Fits right and left / *Thigh circ. taken 15 cm above mid-patella cm						

Code	REF.19034 Mio-S34				
Size	S	M	L	XL	
Circumference cm*	33/36	36/39	39/43	43/48	
Colour	black				
Fits right and left / *Calf circumference					

Code	REF.19037 Malleo-S37				
Size	XS	S	M	L	XL
Circumference cm*	19/21	21/24	24/27	27/30	30/33
Colour	black				
Fits right and left / *Ankle circumference					

MAINTENANCE

Do not bleach No chemical cleaning Do not iron Do not tumble-dry Washing instructions: hand wash at max. 30° C with neutral soap. Dry away from heat sources.

Do not dispose of the product or any of its components into the environment.

INDICATIONS

EPI-S 32 • Chronic pain from bursitis or tendinitis

MANU-S 19 • Aftercare for wrist sprains

• Tendinitis and tenosynovitis

MIO-S 33 • Inflammatory degenerative diseases of the wrist

• Muscular contractions and cramps of the thigh

• Aid to resuming sporting activity after trauma

GENU-S 60 • Gonalgia from bruising

• Early stage arthrosis

• Tendinitis

MIO-S 34 • Muscular contractions and cramps of the calf

• Aid to resuming sporting activity after trauma

MALLEO-S 37 • Bruising

• Tendinopathy of the instep and peronei

KONTRAINDICATIONS

Currently no known

FEATURES AND MATERIALS

• The stretchy tubular support is anatomically designed to fit the joint perfectly

• Breathable

• Medium to high compression

• Comfortable edges

• Composition: 70% polyamide, 30% elastane

NUR MANU-S 19 Thumb hole

ONLY GENU-S 60 Nonslip silicone upper insert

ONLY MALLEO-S 37 Special weave on the heel for a stronger, more durable design

PUTTING ON THE APPLIANCE

EPI-S 32 • Slide the tubular support with the orange label at the top onto the arm

• Position the support so that the label is centred on the back (above the elbow)

MANU-S 19 • Slide the tubular support onto the forearm with the orange label at the top

• Position the support so that the thumb fits into the hole provided.

MIO-S 33 • Position the tubular support onto the thigh with the orange label at the top.

• Position the support so that the label is centred at the front.

GENU-S 60 • Slide the tubular support onto the thigh with the orange label at the top

• Position the support so that the label is centred at the front and the knee is in the middle of the support.

MIO-S 34 • Slide the tubular support onto the calf with the orange label at the top.

• Position the support so that the label is centred.

MALLEO-S 37 • Slide the tubular support with the orange label at the top.

• Position the support so that the labels centred at the front and the heel sits properly in the correct position.

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