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 PLEASE READ THESE INSTRUCTIONS CAREFULLY AND SAVE THEM

Orthosef Line

DECLARATION OF CONFORMITY

As the manufacturer, ORTHOSERVICE AG states under its sole responsibility that this product is a Class I medical device that has been manufactured in accordance with the requirements of Directives 93/42/ EEC and 2007/47/EC. These instructions have been written in conformity with these directives, and their aim is to ensure the safe and proper use of this medical device.

PRECAUTIONS

It is advised that the pressure exerted by the product does not act on parts of the body suffering from wounds, swelling or lumps. The product should not be tightened too firmly so as to avoid the development of pressure points or compression of the nerves or blood vessels underneath. It is advisable to wear the product over a garment, avoiding direct contact with the skin. If in doubt about how to apply, contact an orthopaedic technician. Please read the composition of the product on the inside label carefully. The device should not be worn in the vicinity of open flames or strong electromagnetic fields.

WARNINGS

The product has been designed for the specific indications stated below and must be prescribed by a doctor and fitted by an orthopaedic technician. The orthopaedic technician is responsible for fitting the product and providing information about how to use it safely in accordance with individual requirements. The product must be fitted with the utmost care to ensure its efficacy, tolerability and correct function.

Any changes to the structure or adjustment of the device must be prescribed by a physician and performed by an orthopaedic technician. Under no circumstances should the adjustment made by the physician/orthopaedic technician be altered. The manufacturer is no longer liable if the product is used or adapted inappropriately.

The orthosis is intended to be used only by a single patient; if it is used for more than one patient, the manufacturer accepts no responsibility or liability in accordance with medical device legislation.

Direct contact with the skin may cause reddening or irritation in hypersensitive individuals. In the case of onset of pain, swelling, lumps or any unusual reaction, immediately contact your physician. The orthopaedic efficacy of the product may only be guaranteed when all of its components are used.

SELECTION/SIZE

Code	REF.19032 Epi-S32					
Size	XS	S	M	L	XL	
Circumference cm*	20/23	23/27	27/31	31/35	35/40	
Colour	black					
Fits right and left / *Forearm circumference						

Code	REF.19019 Manu-S19					
Size	XS	S	M	L	XL	
Circumference cm*	14,5/15,5	15,5/16,5	16,5/17,5	17,5/18,5	18,5/20	
Colour	black					
Fits right and left / *Wrist circumference						

Code	REF.19033 Mio-S33					
Size	S	M	L	XL	XXL	
Circumference cm*	45/48	48/52	52/56	56/61	61/67	
Colour	black					
Fits right and left / *Thigh center circumference						

Code	REF.19060 Genu-S60						
Size	XS	S	M	L	XL	XXL	
Circumference cm*	35/38	38/41	41/45	45/49	49/54	54/60	
Colour	black						
Fits right and left / *Thigh circ. taken 15 cm above mid-patella cm							

Code	REF.19034 Mio-S34			
Size	S	M	L	XL
Circumference cm*	33/36	36/39	39/43	43/48
Colour	black			
Fits right and left / *Calf circumference				

Code	REF.19037 Malleo-S37				
Size	XS	S	M	L	XL
Circumference cm*	19/21	21/24	24/27	27/30	30/33
Colour	black				
Fits right and left / *Ankle circumference					

MAINTENANCE

 Do not bleach

 No chemical cleaning

 Do not iron

 Do not tumble-dry

 Washing instructions: hand wash at max. 30° C with neutral soap. Dry away from heat sources.

Do not dispose of the product or any of its components into the environment.

INDICATIONS

EPI-S 32 • Chronic pain from bursitis or tendinitis

MANU-S 19 • Aftercare for wrist sprains

• Tendinitis and tenosynovitis

• Inflammatory degenerative diseases of the wrist

• Muscular contractions and cramps of the thigh

• Aid to resuming sporting activity after trauma

• Gonalgia from bruising

• Early stage arthrosis

• Tendinitis

MIO-S 34 • Muscular contractions and cramps of the calf

• Aid to resuming sporting activity after trauma

MALLEO-S 37 • Bruising

• Tendinopathy of the instep and peronei

CONTRAINDICATIONS

Currently no known

FEATURES AND MATERIALS

• The stretchy tubular support is anatomically designed to fit the joint perfectly

• Breathable

• Medium to high compression

• Comfortable edges

• Composition: 70% polyamide, 30% elastane

• **ONLY MANU-S 19** Thumb hole

• **ONLY GENU-S 60** Nonslip silicone upper insert

• **ONLY MALLEO-S 37** Special weave on the heel for a stronger, more durable design

PUTTING ON THE APPLIANCE

EPI-S 32 • Slide the tubular support with the orange label at the top onto the arm

• Position the support so that the label is centred on the back (above the elbow)

MANU-S 19 • Slide the tubular support onto the forearm with the orange label at the top

• Position the support so that the thumb fits into the hole provided.

MIO-S 33 • Slide the tubular support onto the thigh with the orange label at the top.

• Position the support so that the label is centred at the front.

GENU-S 60 • Slide the tubular support onto the thigh with the orange label at the top

• Position the support so that the label is centred at the front and the knee is in the middle of the support.

MIO-S 34 • Slide the tubular support onto the calf with the orange label at the top.

• Position the support so that the label is centred.

MALLEO-S 37 • Slide the tubular support with the orange label at the top.

• Position the support so that the labels centred at the front and the heel sits properly in the correct position.

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