

heelift

# Suspension boot for heel pressure ulcer prevention and management



**ORTHOSERVICE**  
**RO+TEN**

Take care feel better

# Indications

- **Prevention** of pressure ulcers and all case histories of heel **pressure ulcers or injuries/postoperative** wounds of the heel, including high-risk patients with two or three of the following clinical characteristics:
  - a Braden score of 18 or less
  - prolonged immobilisation
  - diabetes with or without neuropathy

## ADVANTAGES

- Light weight
- Latex-free
- Washable and sanitizable

## CLINICAL EVIDENCE

### **Simulation and Discussion of the Microclimate in Heel Protector Boots**

Evan Call, MS, CSM; Craig Oberg, PhD; Brian Bill, BS Microbiology;  
Chad Mclean Weber State University  
Poster presented at SAWC Fall, 2013

### **An RCT to Determine the Effect of a Heel Elevation Device in Pressure Ulcer Prevention Post-Hip Fracture**

Jeanne Donnelly, PhD, Bsc(hons), MCGI, RGN, ONC; J. Winder PhD, Csci, MIPEM;  
W.G. Kernohan, PhD, BSc CPhys; M. Stevenson.  
Published in Journal of Wound Care, July 2011

### **A Comparison Between the Use of Intravenous Bags and the Heelift Suspension**

Boot to Prevent Pressure Ulcers Isabel Bales, RN/BSN/CWOCN  
Published in Advances in Skin and Wound Care, March 2012

# Why choose heelift?

- It offloads pressure from the heel
- It aids the prevention and management of heel pressure ulcers
- The foot remains in the correct position
- It can be applied very quickly by healthcare professionals
- The boot is patented and validated by clinical evidence

## Three models are available



**Heelift Classic**



**Heelift Glide**



**Heelift AFO**

With a protective rigid rear splint

## BREATHABLE OPEN-CELL POLYURETHANE FOAM INTERIOR



**Smooth**



**Convuluted**



**Ultra**

Available in three finishes:

- **SMOOTH** - The smooth foam has been developed for patients with sensitive skin and helps to prevent the formation of oedema. Designed for patients with diabetes.
- **CONVOLUTED** - The undulating design provides optimum air circulation so the foot feels cooler and more comfortable.
- **ULTRA** - With a liner.


# Features



Toe protection



Soft fastening straps

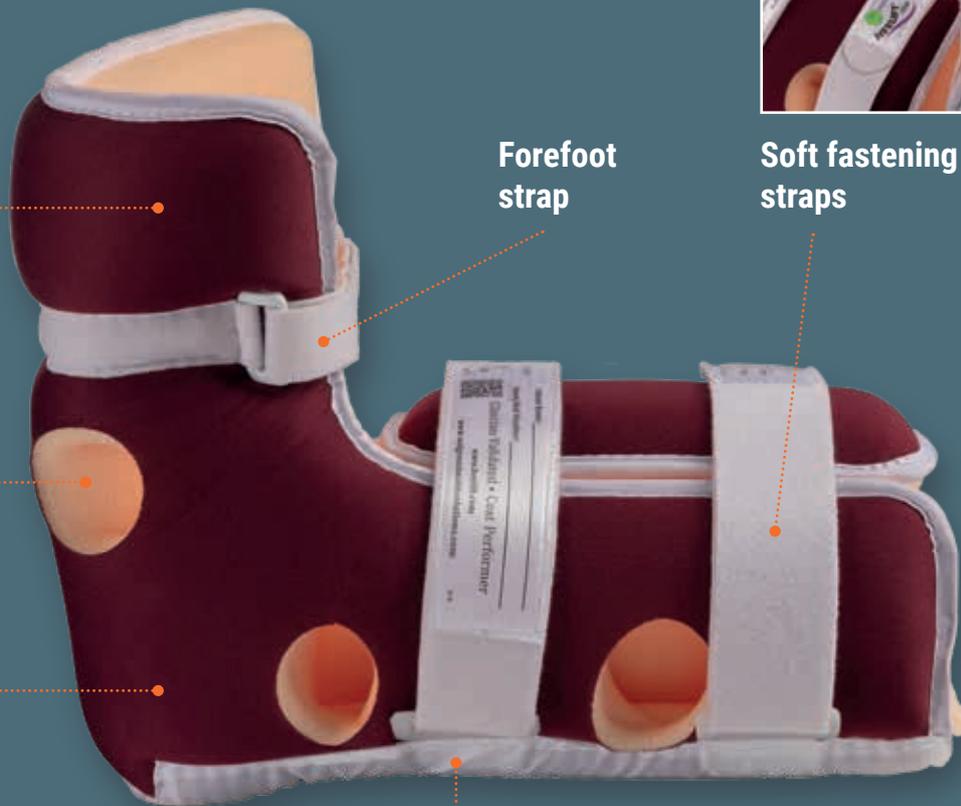

Forefoot strap

Ventilation holes

Durable exterior

Inner rigid rear support and outer material that allows the patient's leg to glide easily across bed sheets



Suspended heel support



Opening for visual inspection of the heel

heelift is supplied  
in **d<sub>2</sub>w** certified  
biodegradable packaging



# Many models, multiple options

FEATURES	HEELIFT CLASSIC		HEELIFT GLIDE			HEELIFT AFO		
	Smooth	Convolved	Smooth	Convolved	Ultra	Smooth	Convolved	Ultra
Suspended heel support	●	●	●	●	●	●	●	●
Opening for visual inspection of the heel	●	●	●	●	●	●	●	●
Durable exterior			●	●	●	●	●	●
Ventilation holes	●	●	●	●	●	●	●	●
Nylon support	●	●	●	●	●	●	●	●
Toe protection			●	●	●	●	●	●
Soft fastening straps	●	●	●	●	●	●	●	●
Forefoot strap			●	●	●	●	●	●
Semi-rigid rear splint						●	●	●
Smooth interior	●		●			●		
Convolved interior		●		●			●	
Inner liner					●			●



Heelift Traction



OPTION: Anti-Rotation Wedge

# heelift

## Suspension boot for heel pressure ulcer prevention and management

MODEL	Size	Calf circumference measured 23 cm above the floor	Code
Heelift Classic Smooth Interior	Petite	6–10 in (15–25 cm)	MHL9041
	Standard	10–15 in (25–38 cm)	MHL9164
	Bariatric	15–23 in (38–58 cm)	MHL7702
Heelift Classic Convoluted Interior	Petite	6–10 in (15–25 cm)	MHL7603
	Standard	10–15 in (25–38 cm)	MHL9348
Heelift Glide Smooth Interior	Petite	6–10 in (15–25 cm)	MHL6477
	Standard	10–15 in (25–38 cm)	MHL6279
	Bariatric	15–23 in (38–58 cm)	MHL6460
Heelift Glide Convoluted Interior	Petite	6–10 in (15–25 cm)	MHL6484
	Standard	10–15 in (25–38 cm)	MHL6453
Heelift Glide Ultra	Petite	6–10 in (15–25 cm)	MHL5500
	Standard	10–15 in (25–38 cm)	MHL5203
	Bariatric	15–23 in (38–58 cm)	MHL5401
Heelift AFO Smooth Interior	Universal	–	MHL8105
Heelift AFO Convoluted Interiors	Universal	–	MHL8204
Heelift AFO Ultra	Universal	–	MHL5302
Heelift Traction	Universal	–	MHL9321
<b>OPTION: Anti-Rotation Wedge</b>	Universal	–	MHL9898



orthoservice.com



Calf circumference measured  
23 cm above the floor



**Walgreen Health Solutions, LLC**  
1316 Sherman Avenue · Evaston, IL 60201  
Phone: (847) 328-9540  
Email: info@walgreenhealthsolutions.com  
www.walgreenhealthsolutions.com  
Made in China



**Emergo Europe BV**  
Prinsessegracht 20  
2514AP The Hague, Netherlands

**Distributed by: ORTHOSERVICE AG**  
Via Milano 7 · CH-6830 Chiasso (TI) · Switzerland  
Tel. 0041 (0) 91 822 00 88 · Fax 0041 (0) 91 822 00 89  
info@orthoservice.com · www.orthoservice.com

Società soggetta a Direzione e Coordinamento (art. 2497bis CC):  
Orthoservice AG (CH) · 6830 Chiasso (TI) · Switzerland



**ORTHO SERVICE**  
**RO+TEN**

Take care feel better