

...and promotes the rehabilitation of the entire joint

The real benefit of a brace working on **3 thrust points** is its deep action: the **thrust on the knee**, combined with the **action of the 4 straps** and the **contralateral lever system**, makes the joint assume a 'position' closer to its physiological one:

- Immediate relief from local pain and relief of joint tendons
- A more natural posture
- Balanced walking
- to the benefit of the entire musculoskeletal system

The comparative Rx data, with and without the brace worn, shows a varus correction of a maximum of 2 degrees, although numerically the data is to be considered small, it is nevertheless significant for all cases of inoperability or of a patient who is not ready for surgery (refusal of surgery).

Considering the clinical case of a major gonarthrosis and therefore, at least radiographically, already of surgical relevance, the correction of 2 degrees is to be considered valid in order to obtain the patient's clinical satisfaction by relieving the most important symptom: pain.

9000P0197N101_0322

genuFEEL OA

Gonarthrosis knee brace

M24014 R
M24014 L

Indications

- Moderate unicompartmental gonarthrosis
- Post-operative cartilage or meniscus surgery

Code	M24014 R · M24014 L						
Size	XS	S	M	L	XL	XXL	3XL
Circumf. 15cm above knee center cm	33/39	39/45	45/52	52/58	58/66	67/74	74/81
Height cm	40	40	40	40	40	40	40
Colour	dark gray						

Right or left

RIGHT = for medial gonarthrosis of the right knee (varus right knee) or for lateral gonarthrosis of the left knee (valgus left knee)

LEFT = for medial gonarthrosis of the left knee (varus left knee) or for lateral gonarthrosis of the right knee (valgus right knee)



orthoservice.com



Headquarter: ORTHOSERVICE AG
Via Milano 7 · CH-6830 Chiasso (TI) · Switzerland
Tel. 0041 (0) 91 822 00 88 · Fax 0041 (0) 91 822 00 89
info@orthoservice.com · www.orthoservice.com

ORTHOSERVICE DEUTSCHLAND GmbH
Flugstraße 8 · D-76532 Baden-Baden · Deutschland
Tel. 0049 (0) 7221 991 39 11 · Fax 0049 (0) 7221 991 39 13
info@orthoservice.de · www.orthoservice.de

ORTHOSERVICE POLSKA Sp. z o.o.
ul. Warszawska 416a · 42-209 Częstochowa · Polska
Tel. +48 (0) 34 340 13 10 · (NIP): 9492246785
info@orthoservice.pl · www.orthoservice.pl

RO+TEN S.r.l.
Sede legale: Via Marco De Marchi 7 · I-20121 Milano (MI) · Italia
Sede operativa e amministrativa:
Via Comasina 111 · I-20843 Verano Brianza (MB) · Italia
Tel. 0039 039 601 40 94 · Fax 0039 039 601 42 34
info@roplusten.com · www.roplusten.com



genuFEEL OA

Gonarthrosis knee brace



Time, a key factor

Relieving pain improves patients' quality of life:

- inoperable
- awaiting surgery
- at an age not yet suitable for surgical treatment

this is the real goal. **GenuFeel OA** alleviates the side effects of degenerative gonarthrosis.




Fastening, hooking the straps, **lowering** the levers: a sequence of three simple gestures, made possible by innovative design, sophisticated materials and increasingly functional details.

The only patented system that takes less than 1 minute...



Knee unloading mechanism

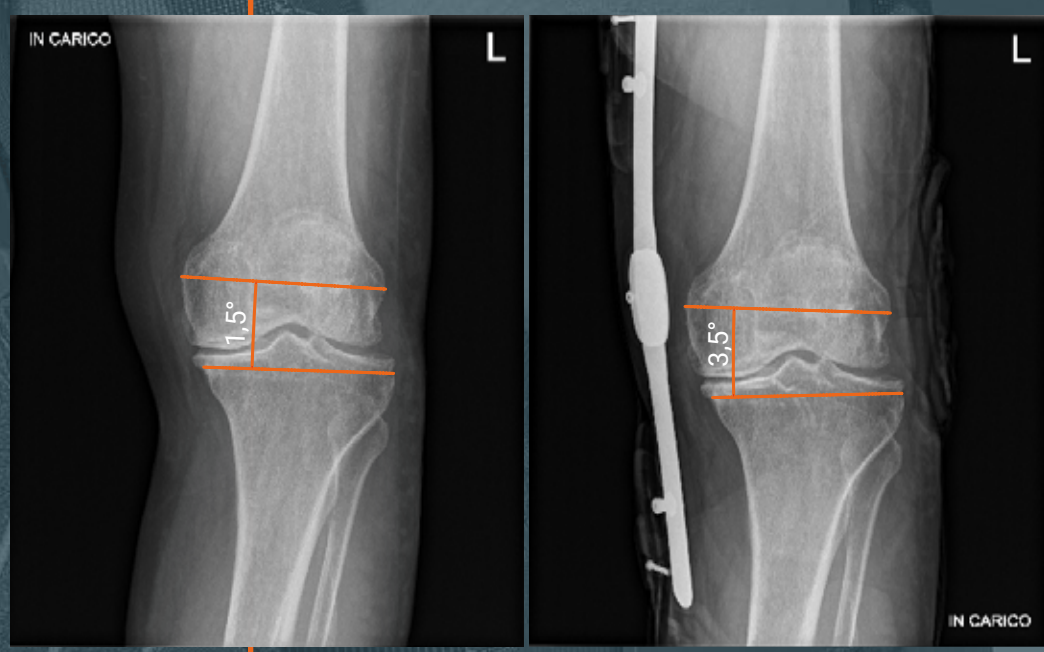

TOTAL COMFORT

on the skin, thanks to the stretchy, breathable fabric, made with no seams and no gluing, and to the padding incorporated in the pressure zones



EFFECTIVE DISTRIBUTION OF THRUST PRESSURE

thanks to the new ring which also features an internal protective padding, in gel and removable



FLEXION-EXTENSION ADJUSTMENT

with simplified joint



EASY TENSIONING OF THE STRAPS

thanks to **low-profile levers** that only need a light pressure from the palm of the hand



EASY AND QUICK FITTING

front opening and finger pockets that facilitate the use of the Velcro® fasteners



IMMEDIATE DETERMINATION OF STRAP TENSION,

for possible adjustments: **graduated scale**



INTUITIVE PRE-POSITIONING AND POSITIONING

of the straps: **coloured loops**

SECURE FIT TO THE LIMB AT ALL TIMES,

thanks to the anatomical shape, the technopolymer stabilising **valves**, the steel **rod** and the flexible polymer **splints**



The gonarthrosis knee brace that works when you need it