REF.24610 - HERNIX 10

REF.24642 - HERNIX 42

REF.24643A - HERNIX 43A

REF.24643C - HERNIX 43C

REF.24649 - HERNIX 49

REF.24690 - HERNIX 90

REF.24645 - HERNIX 45

REF.24646 - HERNIX 46

REF.24647 - HERNIX 47

REF.24699W - HERNIX P

OPTION: REF.24699W
Doppelpack Pelotten für Hernix, Damen
Couple of pads for Hernix, female
Paire de pelotes pour Hernix, femme
Para pelotek dla Hernix, kobieta
Coppia di pelote per Hernix, donna

REF.24699M - HERNIX P

OPTION: REF.24699M
Doppelpack Pelotten für Hernix, Herren
Couple of pads for Hernix, men
Paire de pelotes pour Hernix, homme
Para pelotek dla Hernix, mężczyzna
Coppia di pelote per Hernix, uomo

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PLEASE READ THESE INSTRUCTIONS CAREFULLY AND KEEP THEM

Sanitary underwear

REF.24610 Classic hernia brief

REF.24643A Two-tone containment hernia brief

REF.24649 Hernia boxer briefs

REF.24645 Post op containment briefs

REF.24647 Jockstrap

REF.24642 Containment hernia brief

REF.24643C Two-tone containment hernia brief

REF.24690 Female hernia brief

REF.24646 Sports containment briefs

DECLARATION OF CONFORMITY

As the manufacturer, ORTHOSERVICE AG declares, under its sole responsibility, that this is a class I medical device that has been manufactured and assessed, according to the EU Regulation 2017/745 (MDR). These instructions have been drawn up in application of the aforementioned Regulation. They are intended to ensure an adequate and safe use of the medical device.

SAFETY PRECAUTIONS

We recommend that the pressure exerted by the device does not act on parts of the body with wounds, swelling, or welts. It is advisable to not over tighten the device so as to avoid generating areas of excessive local pressure or the compression of underlying nerves and/or blood vessels. If in doubt on how to apply the device, contact a doctor, physiotherapist, or an orthopedic technician. Please report the composition of the product on the inside label carefully, do NOT use the product if the patient has known intolerances or allergies to one or more components in the product. We recommend to not wear the device in the proximity of free flames. Do not apply in direct contact with open wounds.

WARNINGS

It is advisable that the device, designed for the specific indications as set out below, be prescribed by a doctor or a physiotherapist and applied by an orthopedic technician, according to the specific needs of the patient. To ensure effectiveness, tolerability, and proper functioning, the application must be carried out with the utmost care. Never alter the adjustment made by the doctor/physiotherapist/ or orthopedic technician. The responsibilities of the manufacturer shall lapse in case of inappropriate use or adaptation. The orthosis is made and to be used by only one patient. Should the orthosis be inappropriately used, the manufacturer declines all responsibility, as provided by the regulation for medical devices. In hypersensitive individuals, direct contact with the skin may cause redness and irritation. In case of pain, swelling, welts, or any other adverse reaction, contact your doctor immediately, and in case of a particularly severe adverse event, report the incident to the manufacturer and the competent authority of your country. The effectiveness of the orthopedic product is only guaranteed when all the components are properly in use. Contact an orthopaedic technician for the replacement of used parts

GENERAL INTRODUCTION

Orthoservice Ro+Ten Hernix line of briefs: from hernia briefs to sports briefs, a complete line dedicated to the different needs of abdominal and groin containment.

HERNIA BRIEFS (ref.24649, 24610, 24690, 24643A, 24643C, 24642)

Hernix hernia briefs are designed to contain and support the abdominal tissues and are indicated in cases of starting hernias and reducible inguinal hernias. They can also be used in the postop phase and as a prevention against recurrence. They are designed to be fitted with hernia compression pads.

The exclusive cotton-based, supportive, thin and soft fabrics, combined with appropriately shaped reinforcements, guarantee effective containment of the hernia and comfort in daily use.

CONTAINMENT BRIEFS

Hernix containment briefs have the function of containing and supporting the abdominal tissues and/or genital region:

- Postop briefs (ref.24645) are indicated after abdominal surgery and in case of abdominal relaxation.
- Sports briefs (ref.24646) and jockstraps (ref.24647) are indicated during physical and sporting activity and in all cases where good support of the genital region is required.

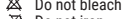
All briefs are made from exclusive cotton-based fabrics, which guarantee comfort, hygiene and breathability: they are suitable for all-day wear.

SELECTION/SIZE

Code	REF.24610 - REF.24642 - REF.24643A - REF.24643C - REF.24649 - REF.24690 - REF.24645 REF.24646 - REF.24647							
Size	1	2	3	4	5	6	7	8
Hip circumf.	83/87	88/92	93/97	98/103	104/109	110/115	116/122	123/130

Colour: REF.24610 light grey - **REF.24642** anthracite grey - **REF.24643A** anthracite/ light grey - **REF.24643C** anthracite/ light grey - **REF.24649** anthracite grey - **REF.24690** nude - **REF.24645** anthracite grey - **REF.24646** anthracite grey - **REF.24647** anthracite grey


MAINTENANCE

Do not bleach

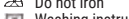No chemical cleaning

Do not iron

No do tumble-dry

Washing instructions: Wash on a delicate cycle at a maximum of 30°. Leave to dry away from heat sources.

Do not dispose of the device or any of its components in the environment.

INDICATIONS

REF.24649 - REF.24610 - REF.24642 - REF.24643A - REF.24643C

- Containment for starting hernias and reducible inguinal hernias
- Pre-op (with the insertion of optional hernia compression pads)
- Post-op
- Recurrence prevention

REF.24690

- Inguinal hernias, ptosis, and prolapse containment
- Post-op
- Hernia recurrence prevention

REF.24645

- Post-abdominal surgery or in case of tissue relaxation for abdominal wall containment
- Genital support

REF.24646

- Preventive action genital support during work activities and sports

REF.24647

- Preventive action genital support during work activities and sports

CONTRAINDICATIONS

Do not use in case of incarcerated or strangulated hernias, at an advanced stage of inflammation or irreducible.

CHARACTERISTICS AND MATERIALS

REF.24610

- Very supportive cotton fabric structure
- Reinforced and shaped internal structure
- Internal under-thigh pullers
- Soft, stretchable front with side openings
- Fitted for optional hernia pads(ref.24699M)

REF.24649 - REF.24642 - REF.24643A - REF.24643C

- Hypoallergenic, thin, soft and comfortable cotton fabric structure
- Reinforced and shaped inner structure (adapted for boxer brief fit Only REF. 24649)
- Under-thigh inner pullers, with calibrated traction
- Closed and shaped front, lined with soft cotton jersey
- Fitted for optional hernia pads (ref.24699M)

REF.24690

- Very supportive, hypoallergenic, soft and comfortable cotton fabric structure.
- Reinforced and shaped internal structure for correct support of the abdominal wall
- Soft finishing on the leg area for maximum comfort
- Fitted for optional hernia pads (ref.24699W)

REF.24645

- Soft, thin, supportive cotton fabric structure
- Reinforced and shaped inner structure for effective containment of the abdominal wall and support of the genital region
- Closed, soft, highly stretchable waistband

REF.24646

- Medium containment cotton fabric structure, thin and soft
- Closed front, with calibrated containment for correct containment of the genital region

REF.24647

- Tailored cotton fabric front for correct containment of the genital region
- Soft elastic under thigh
- Soft finish crotch

APPLICATION

1 Wear like normal underwear

2 Make sure to exclude genitals from the containment area

3 (ONLY HERNIA models, REF. 24649, 24610, 24690, 24643A, 24643C, 24642) En cas d'utilisation des

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If compression pads are used, place them inside the fitted pockets.

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