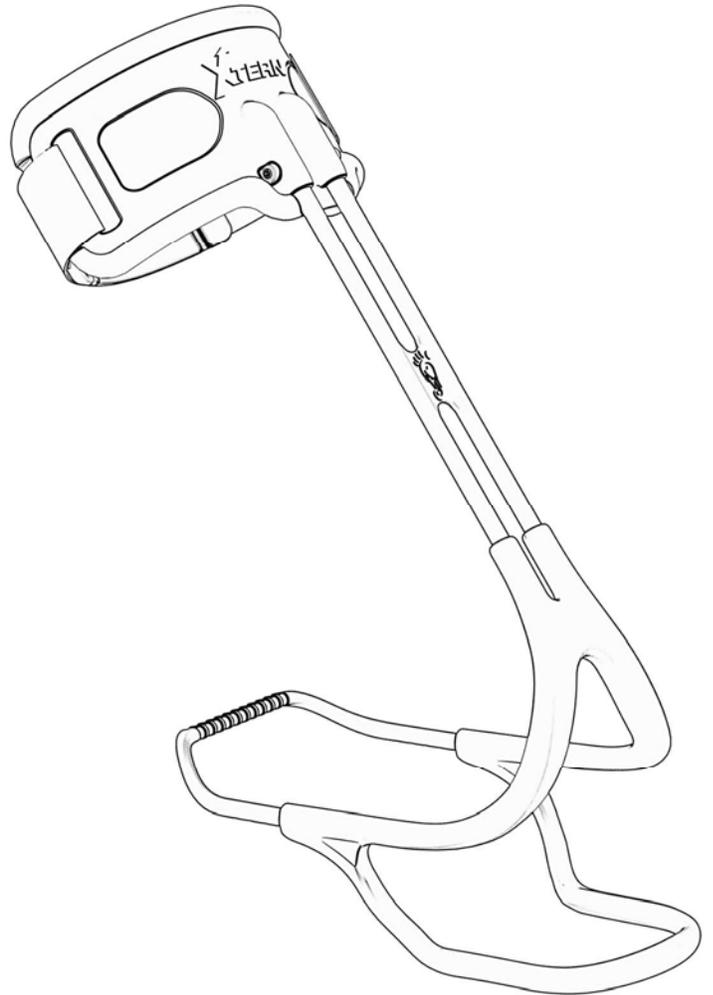
XTERN[®] CLASSIC+

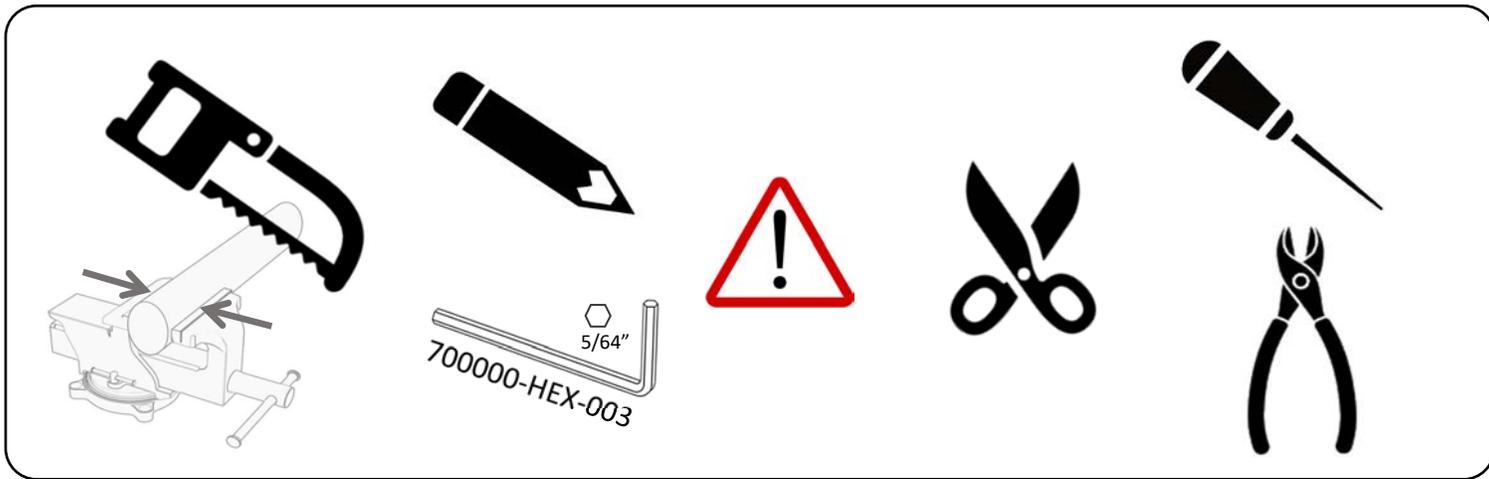
 turbomedorthotics.com
 turbomedusa.com
 info@turbomedorthotics.com

  TurboMed Orthotics Inc.
240 Sydney (101),
St-Augustin-de-Desmaures , QC
G3A 0P2 CANADA T: 1-888-778-8726

U.S. Patent No. : US 8,529,484, USD 804,044, USD 875,960, US 11,471,313
Canadian Patent & Industrial design No. : CA 2 692 534, CA 2 900 058, CA 168 202
European Patent & Community design No. : EP 2 533 734, 003075654-01/02,
GB 90030756540001, 0002, 008883326-0001, GB 6197836
Patents pending worldwide: PCT/CA2018/051062, PCT/CA2022/051826

WARNING!

Wear appropriate personal protective equipment, depending on the nature of the tool and the associated hazards. Keeping tools clean and free from grease, oil, or other substances that may affect grip or performance. Applying the appropriate force and using proper techniques while using hand tools to prevent slips, loss of control, accidental cuts, or injuries. You may use fixtures to hold the workpiece securely instead of relying on handholding. Following guidelines for safe operation, including positioning hands and fingers away from cutting edges or moving parts, and avoiding awkward or unstable positions while operating the tool. Keeping tools out of reach of children or unauthorized individuals. Maintaining a clean and well-organized work area to reduce the risk of accidents or tripping hazards. Regularly inspecting tools for damage, wear, or defects, and removing damaged tools from use until repaired or replaced.

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


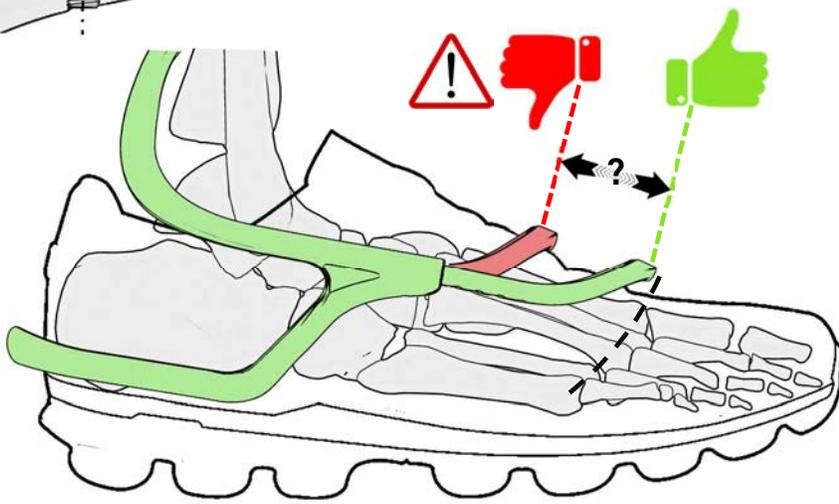
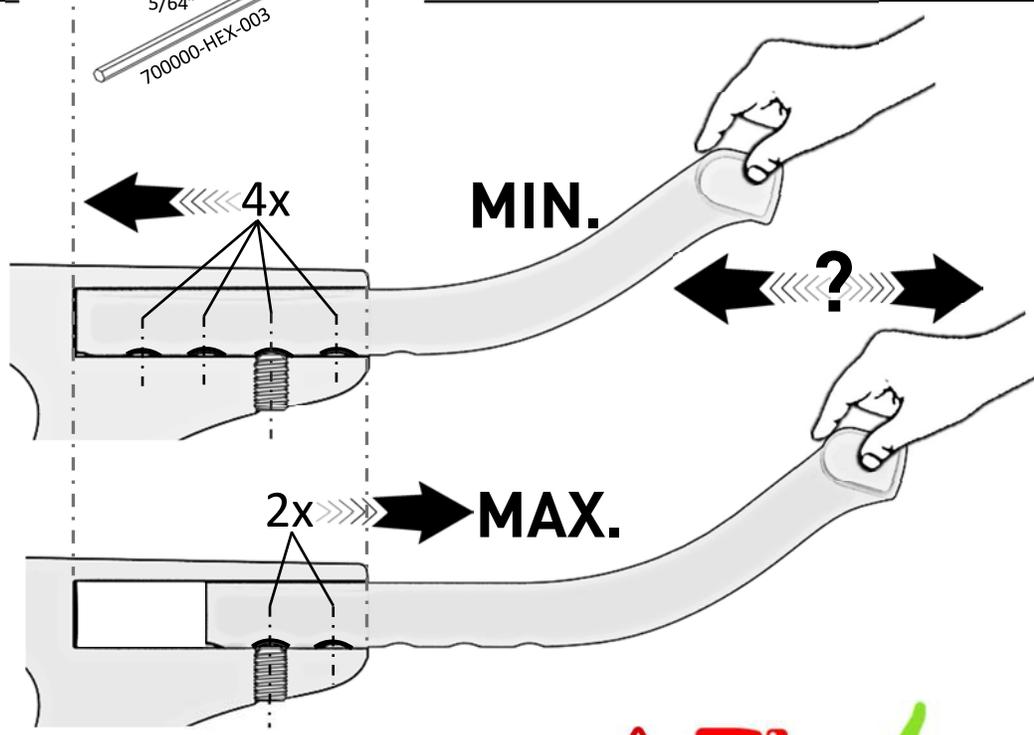
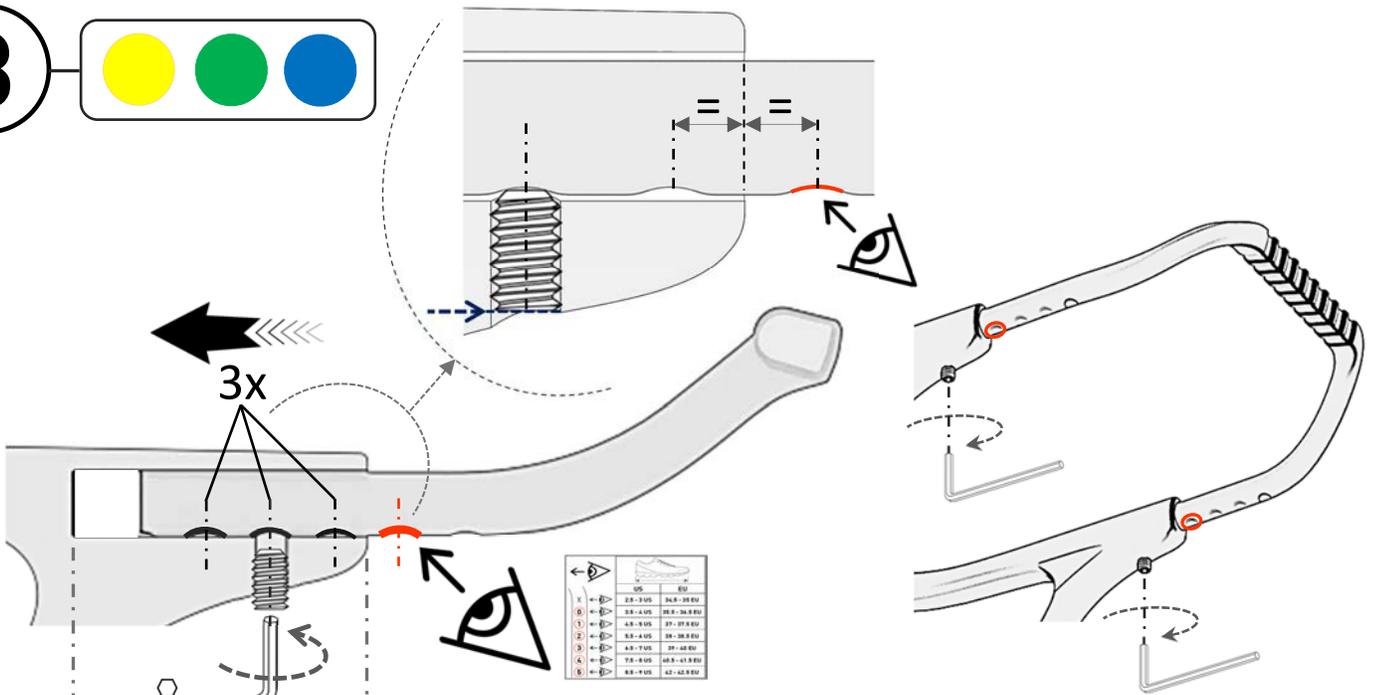
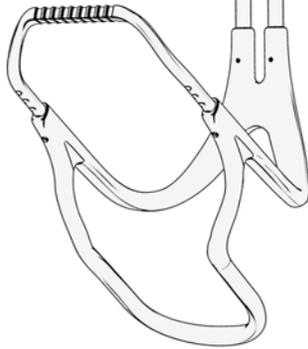
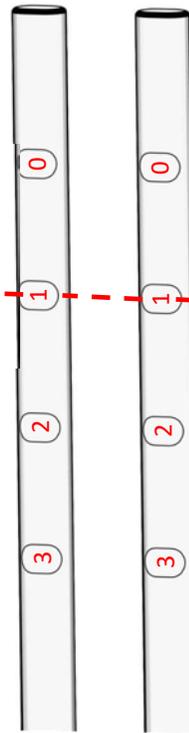
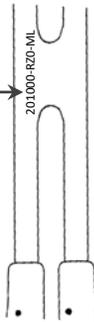
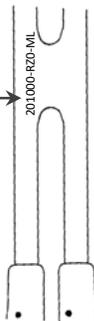
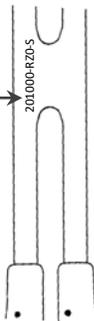
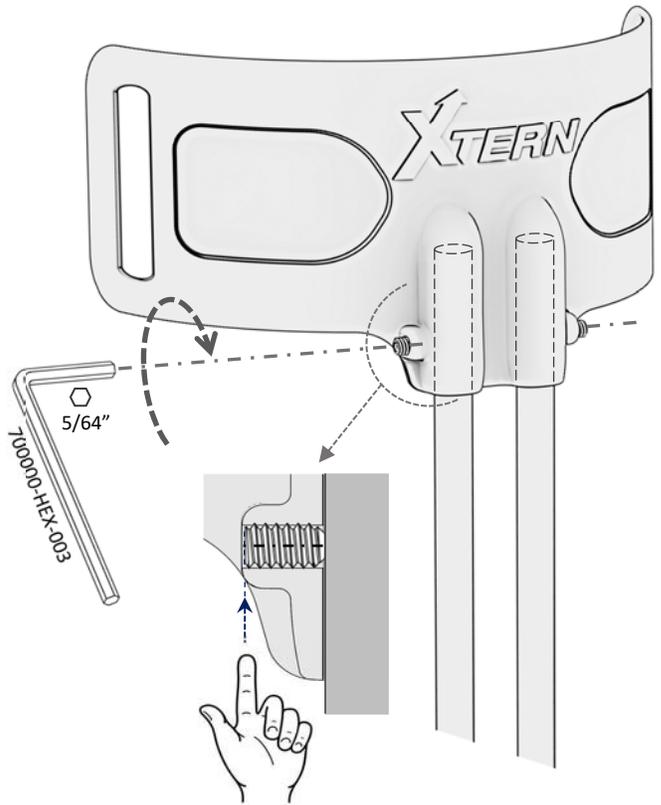
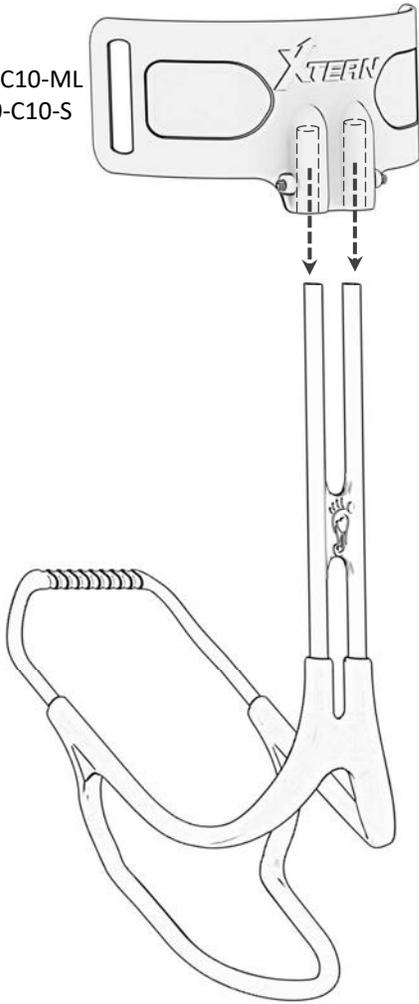
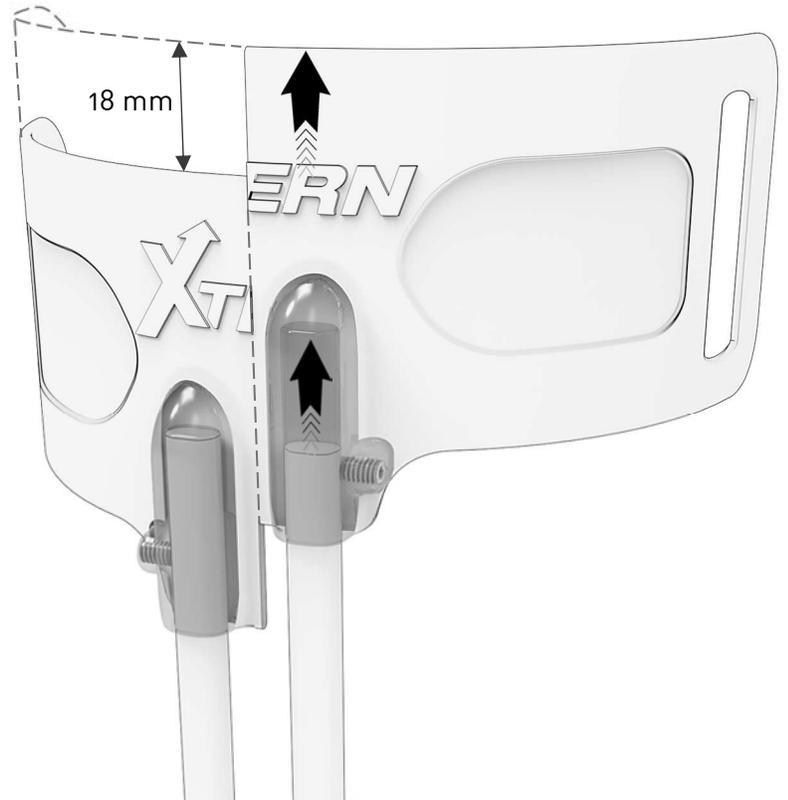
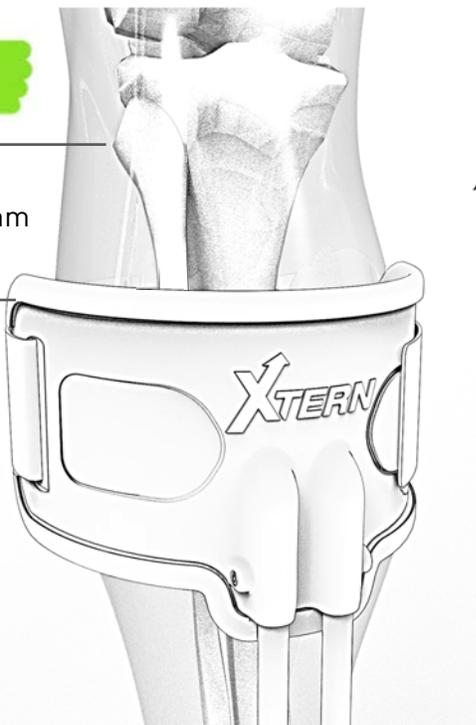
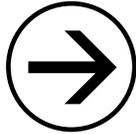
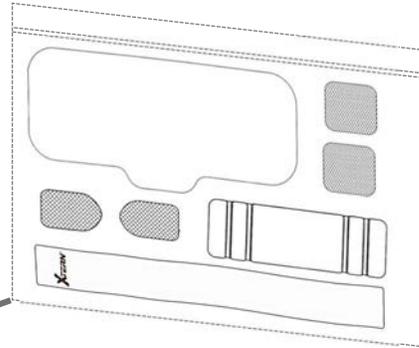
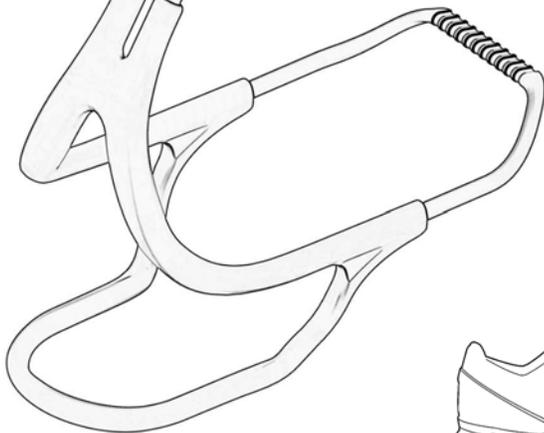
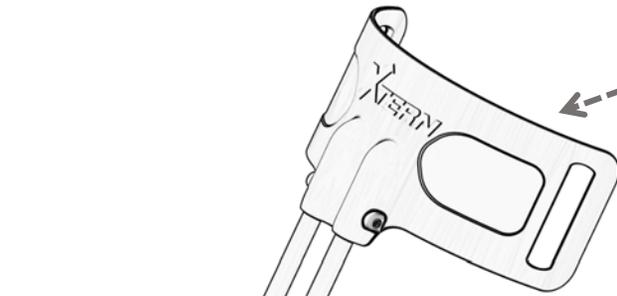
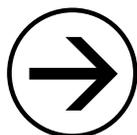
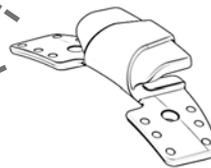
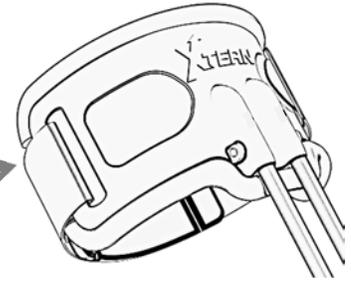
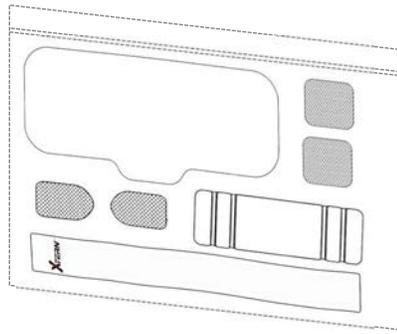
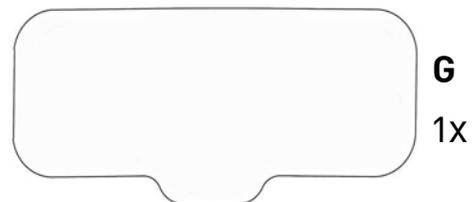
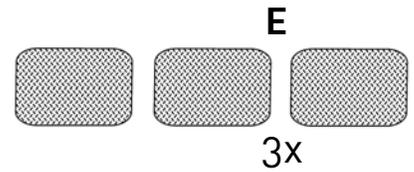
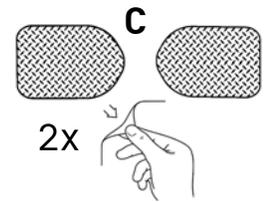
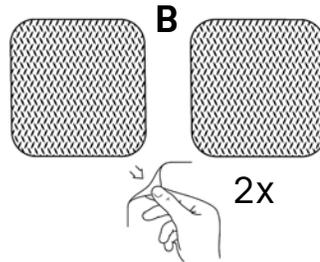
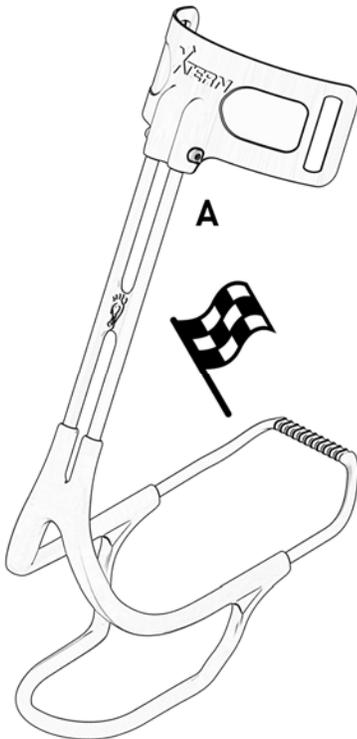
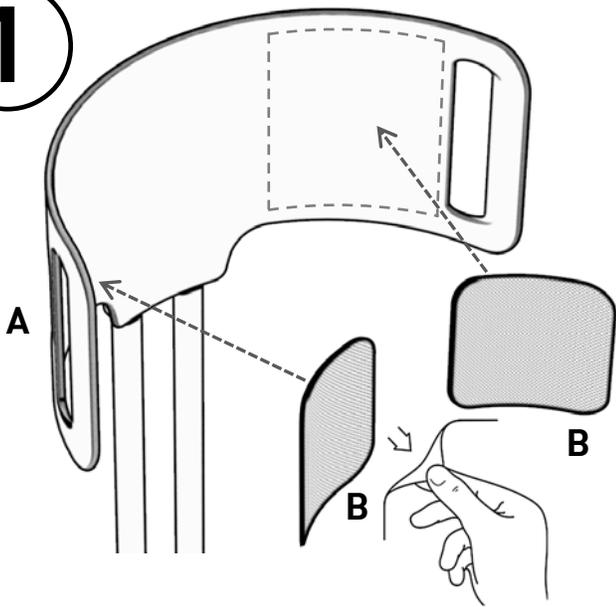
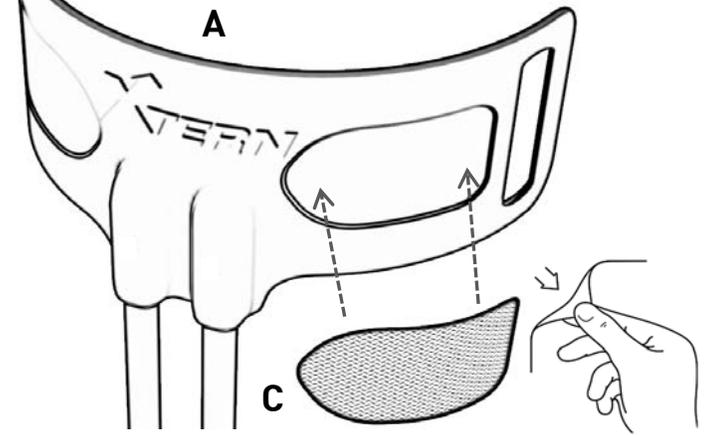
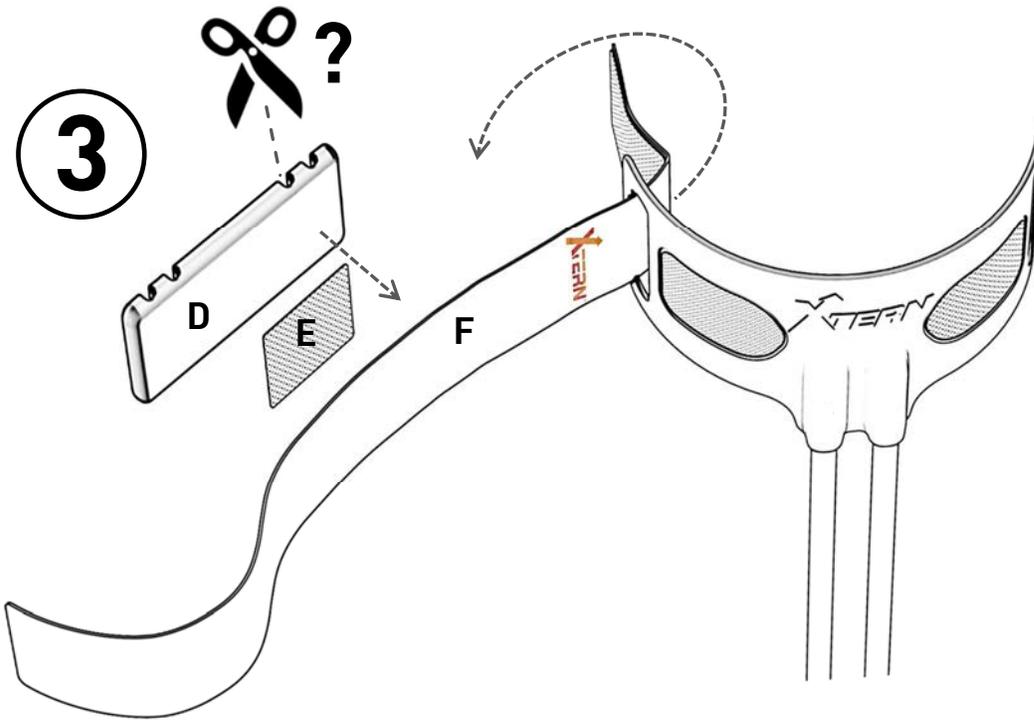
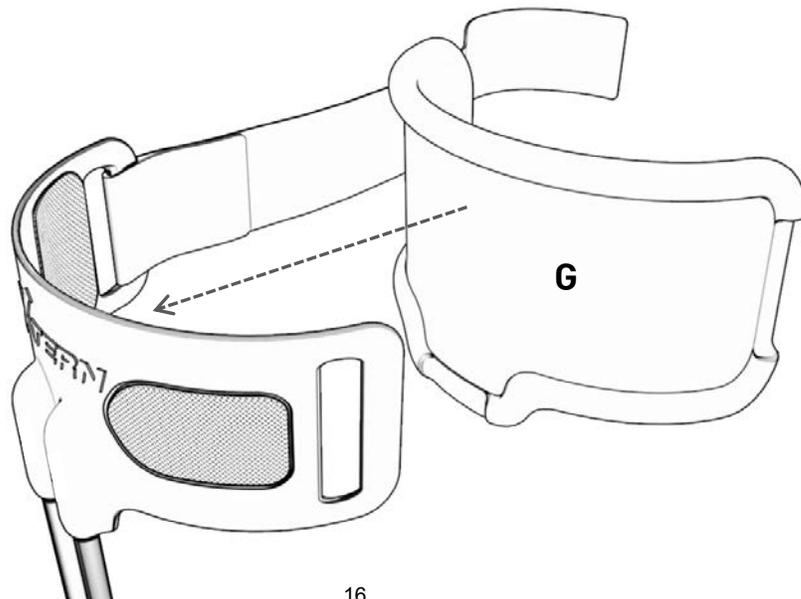
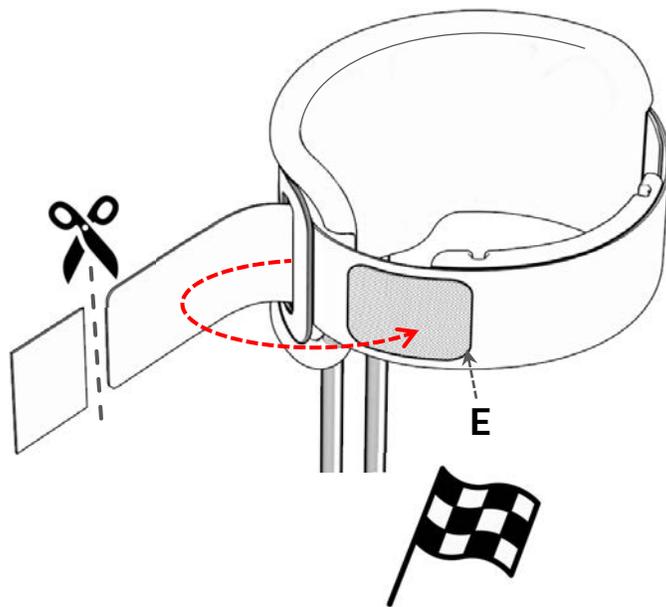
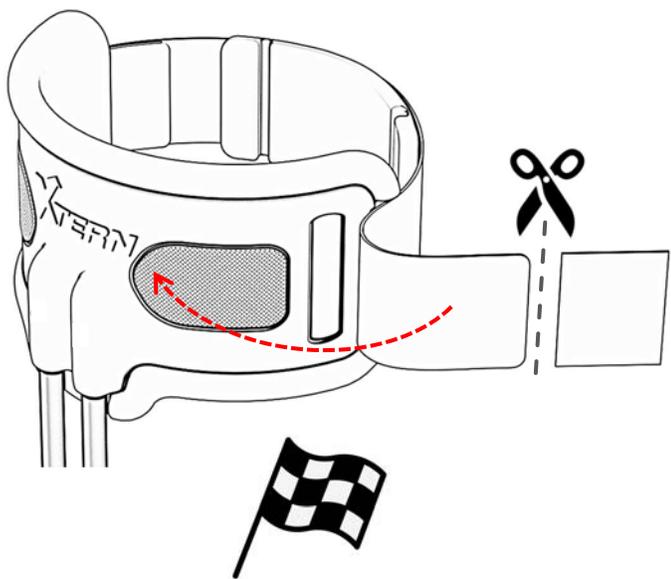
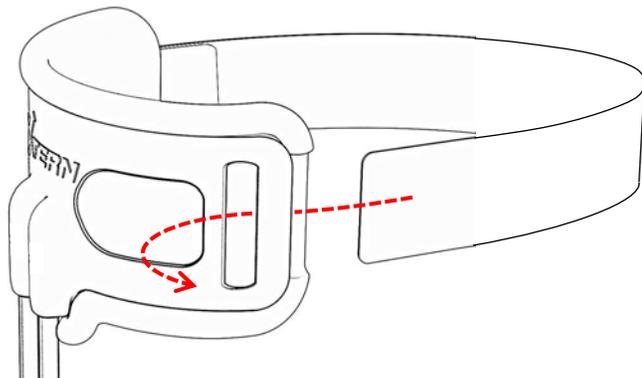
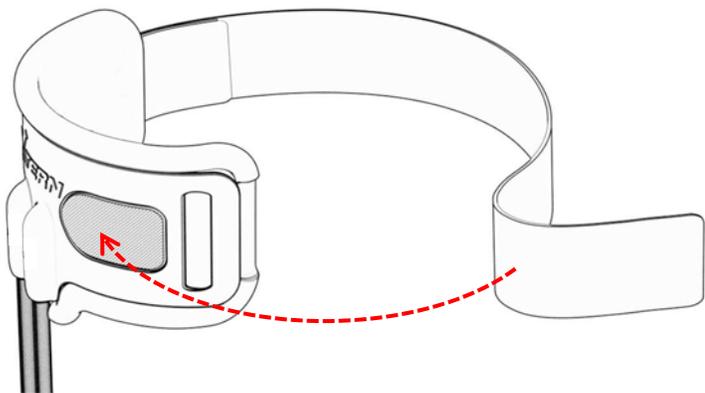
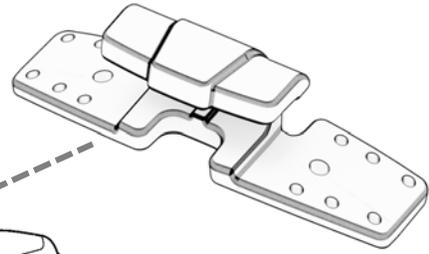
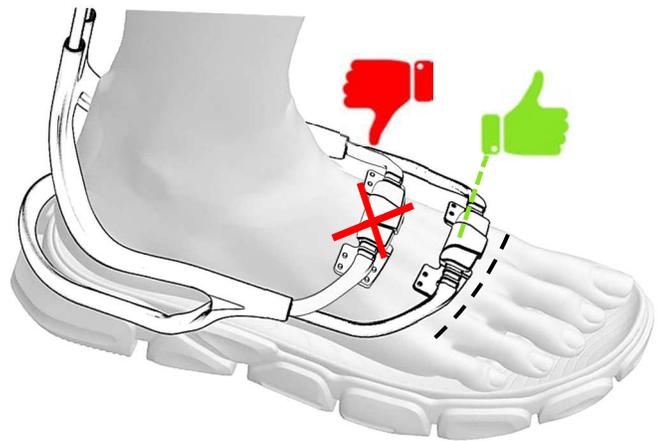
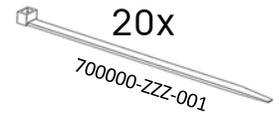
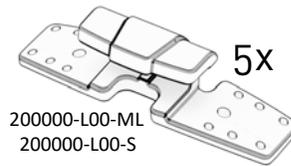
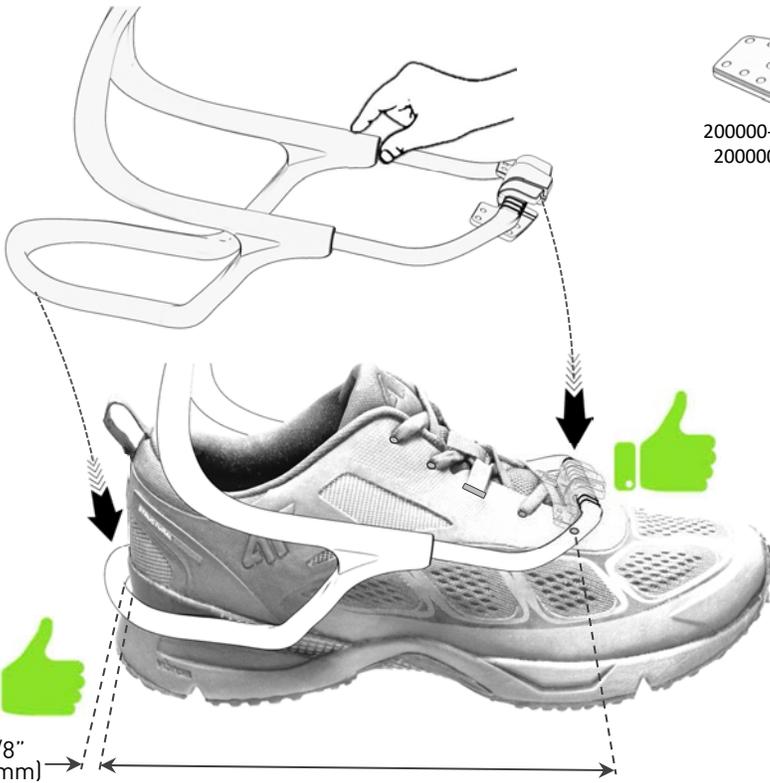
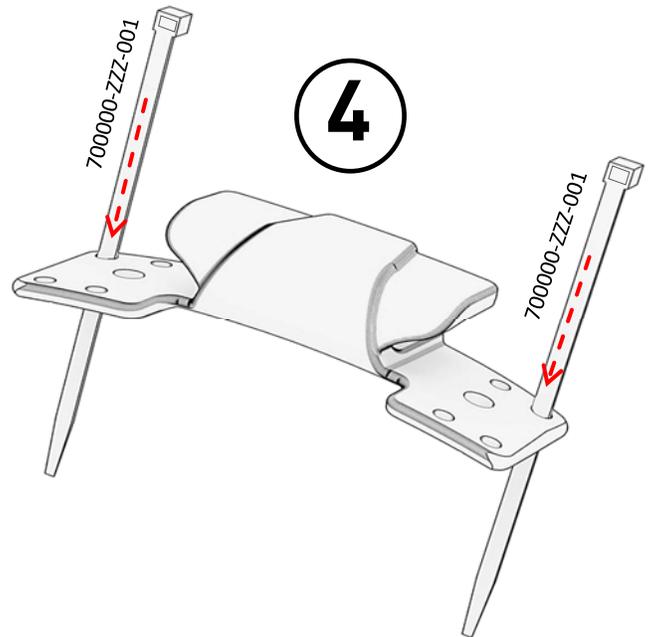
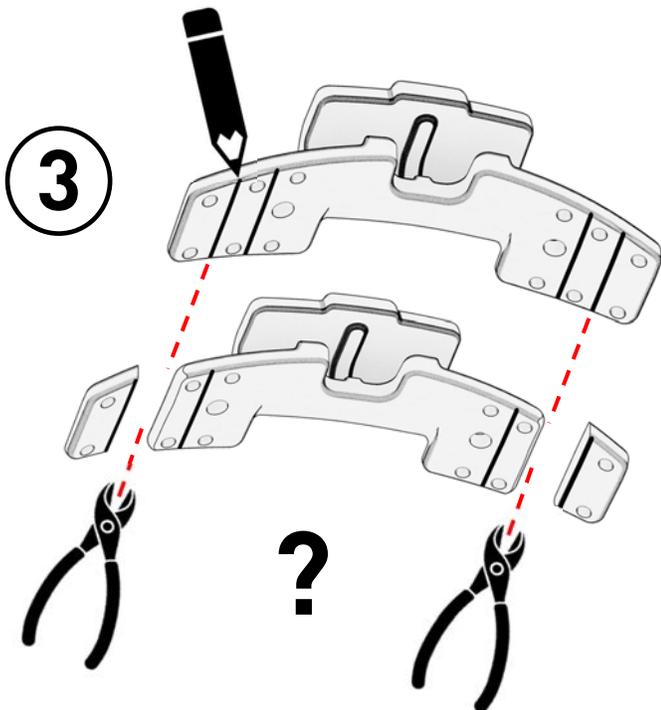
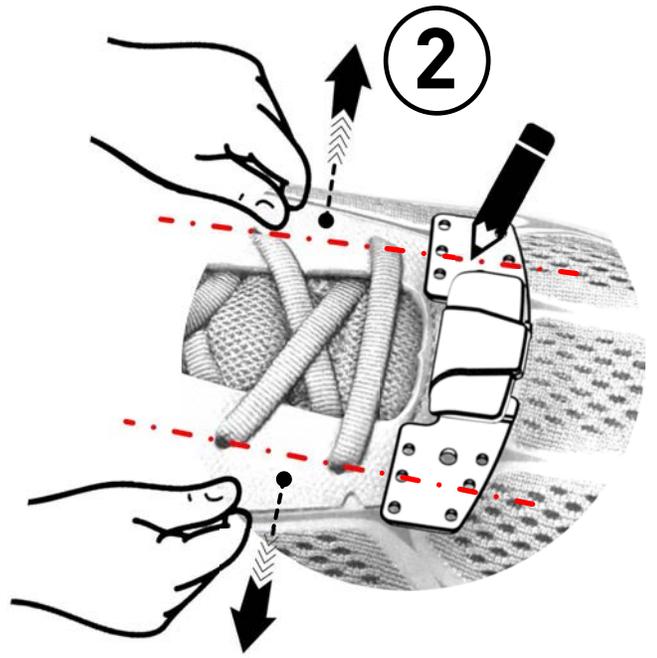
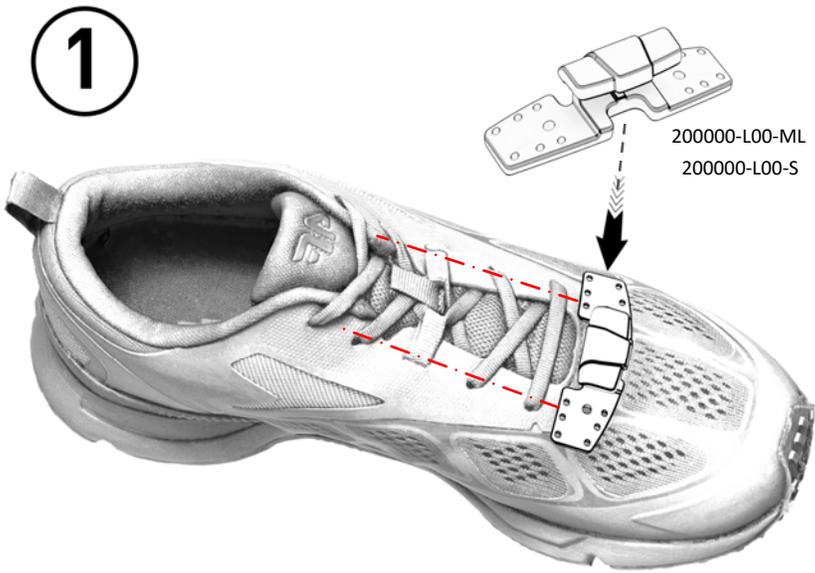
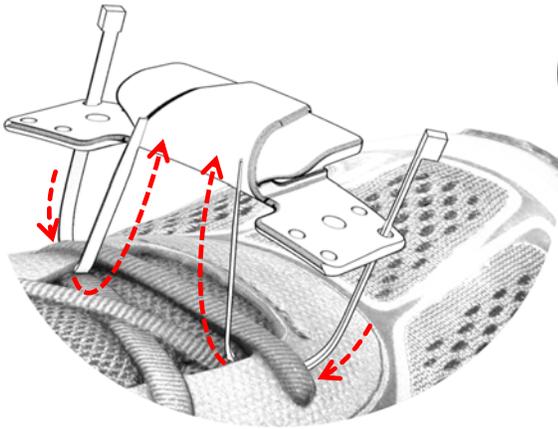
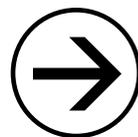
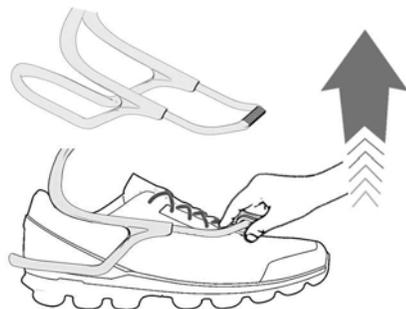
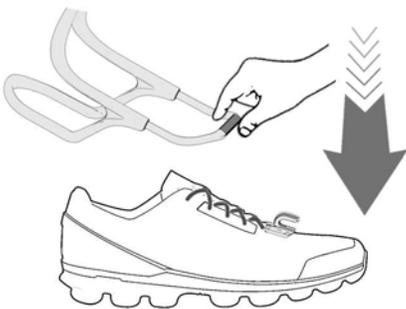
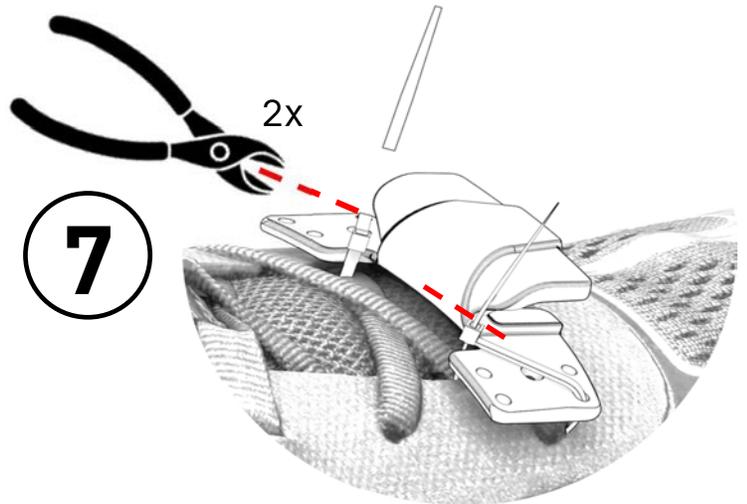
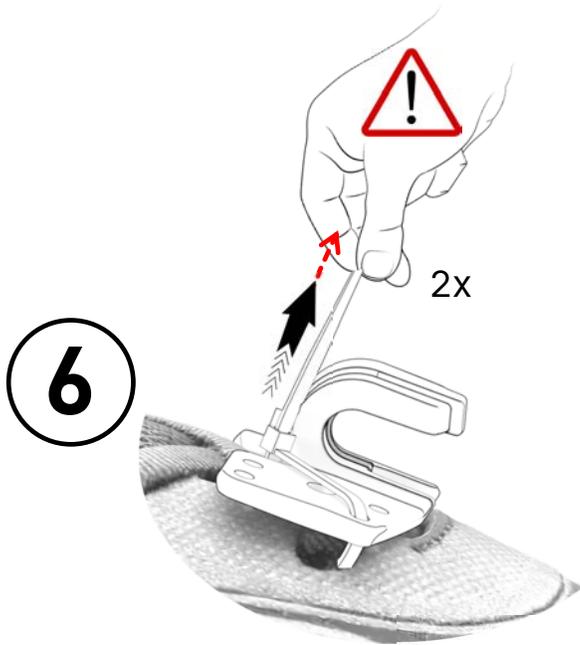
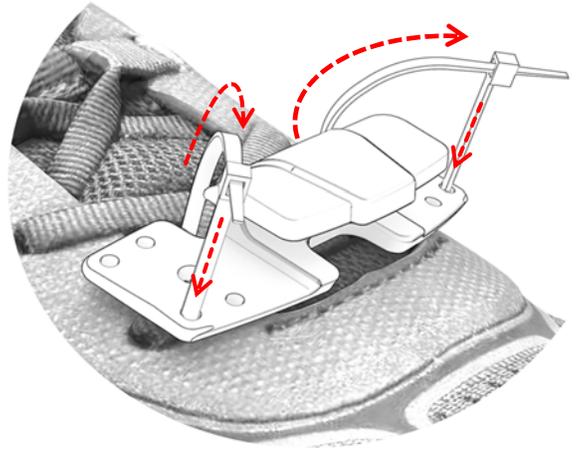
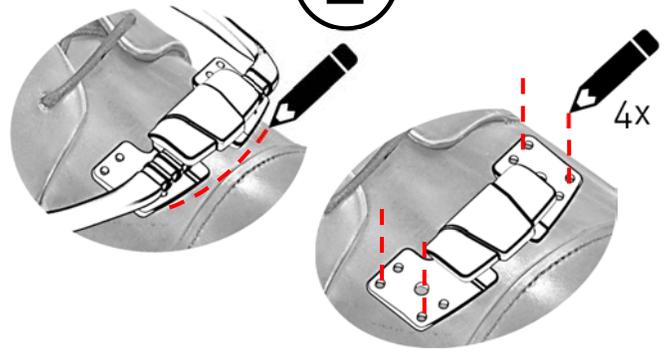
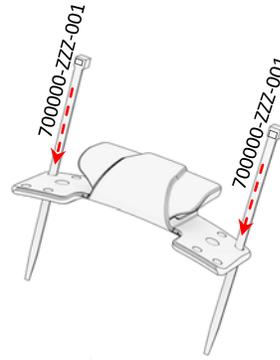
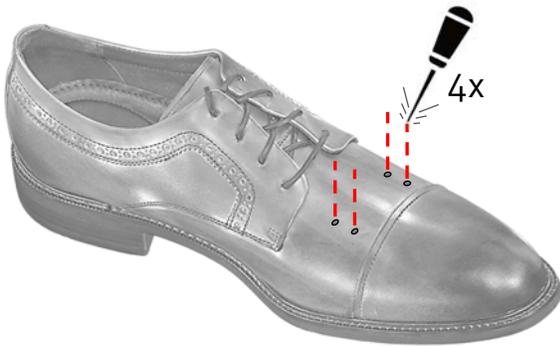
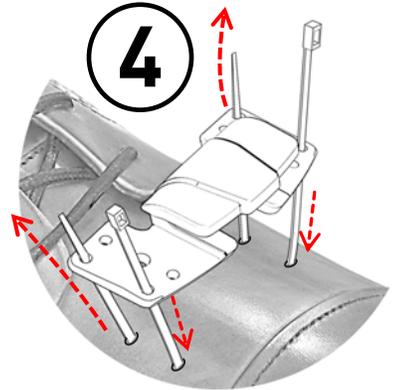
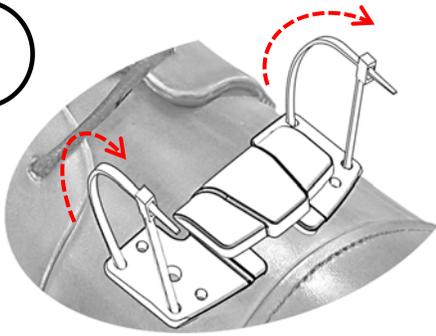
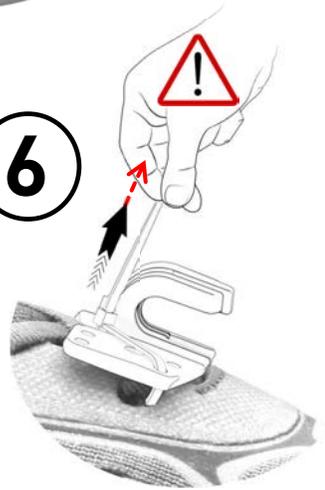
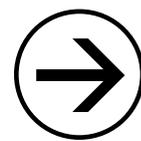
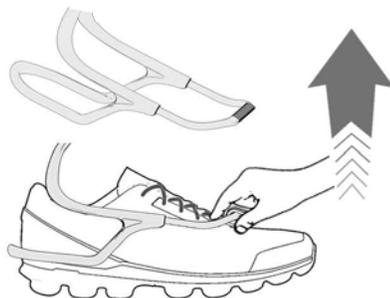
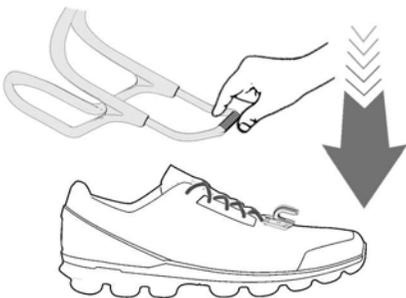
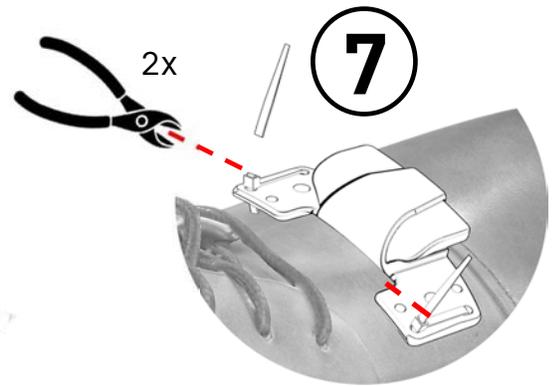
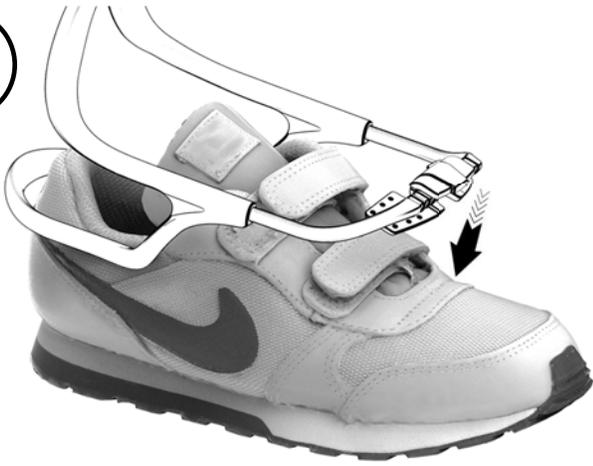
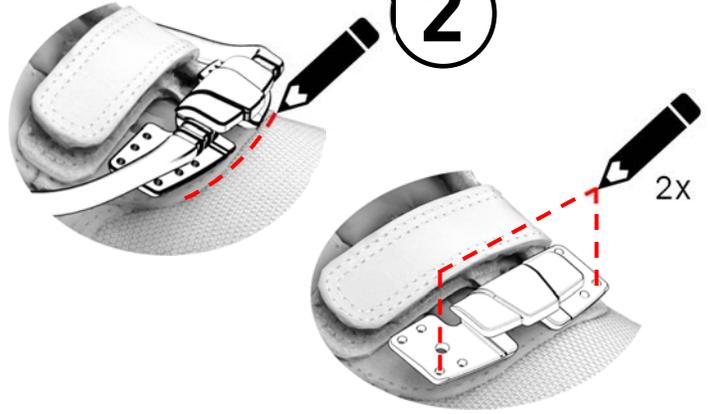
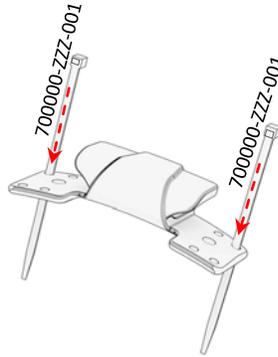
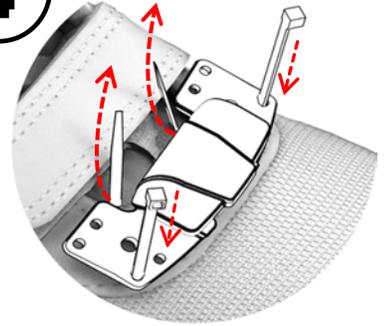
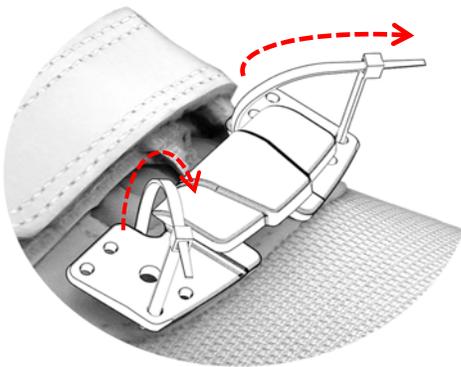
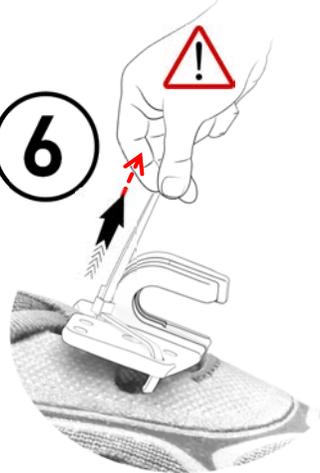
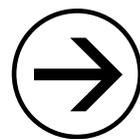
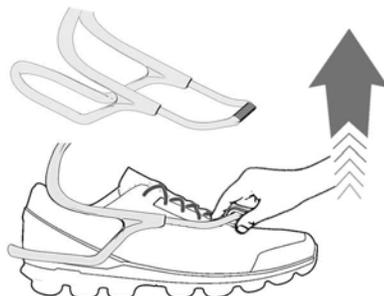
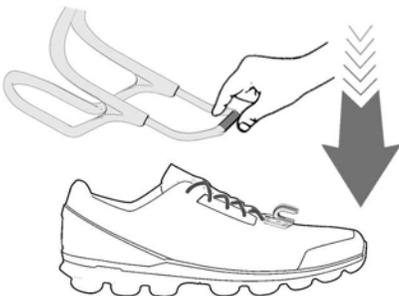
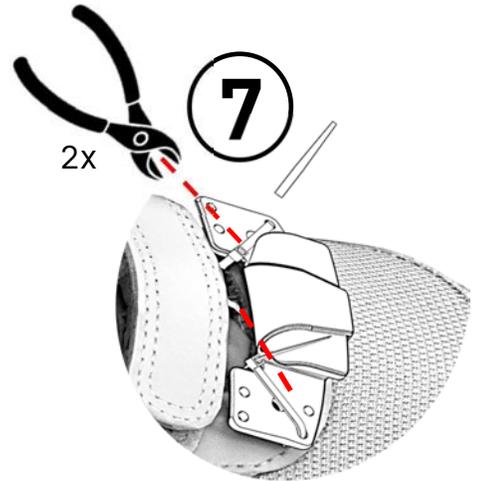
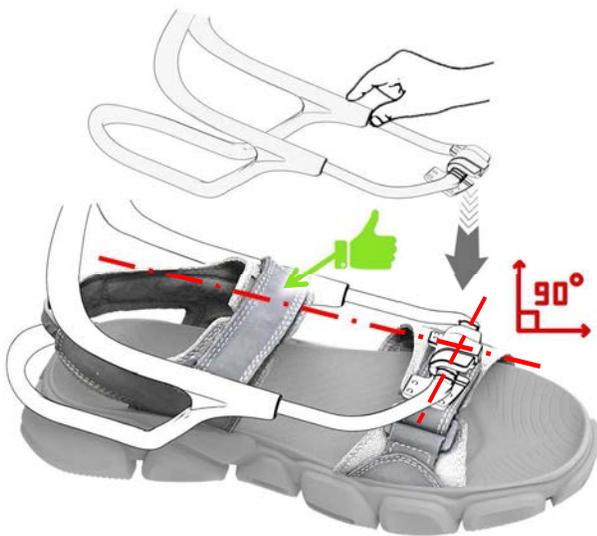
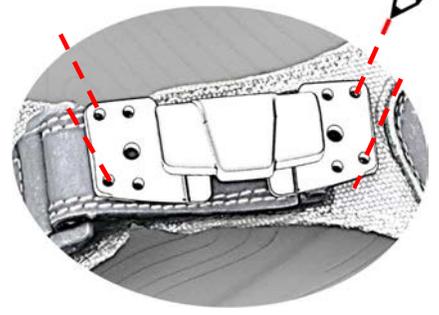
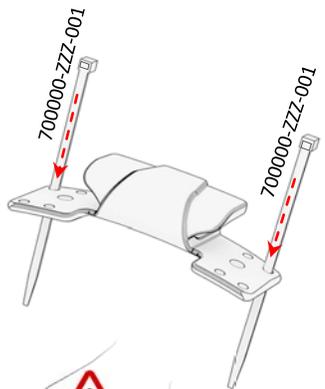
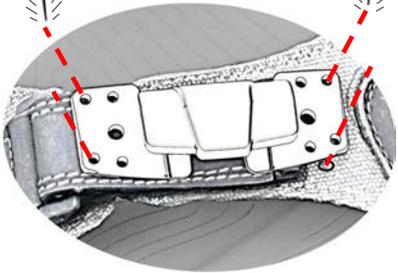
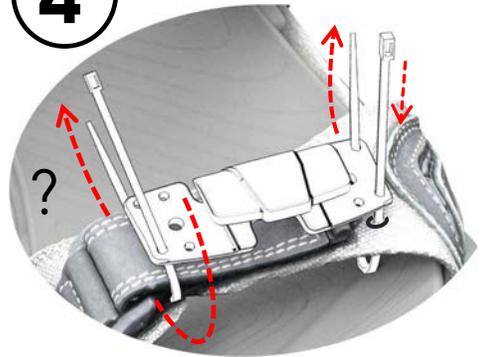
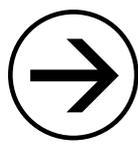
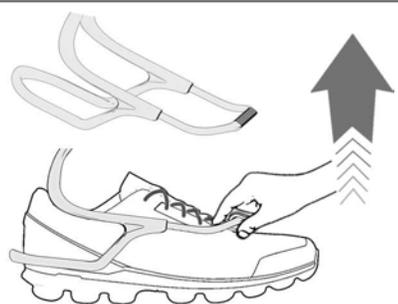
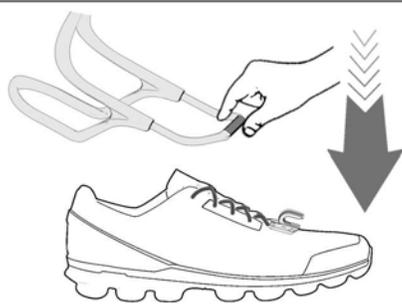
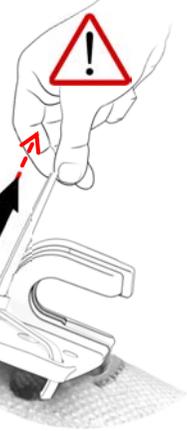
p.24

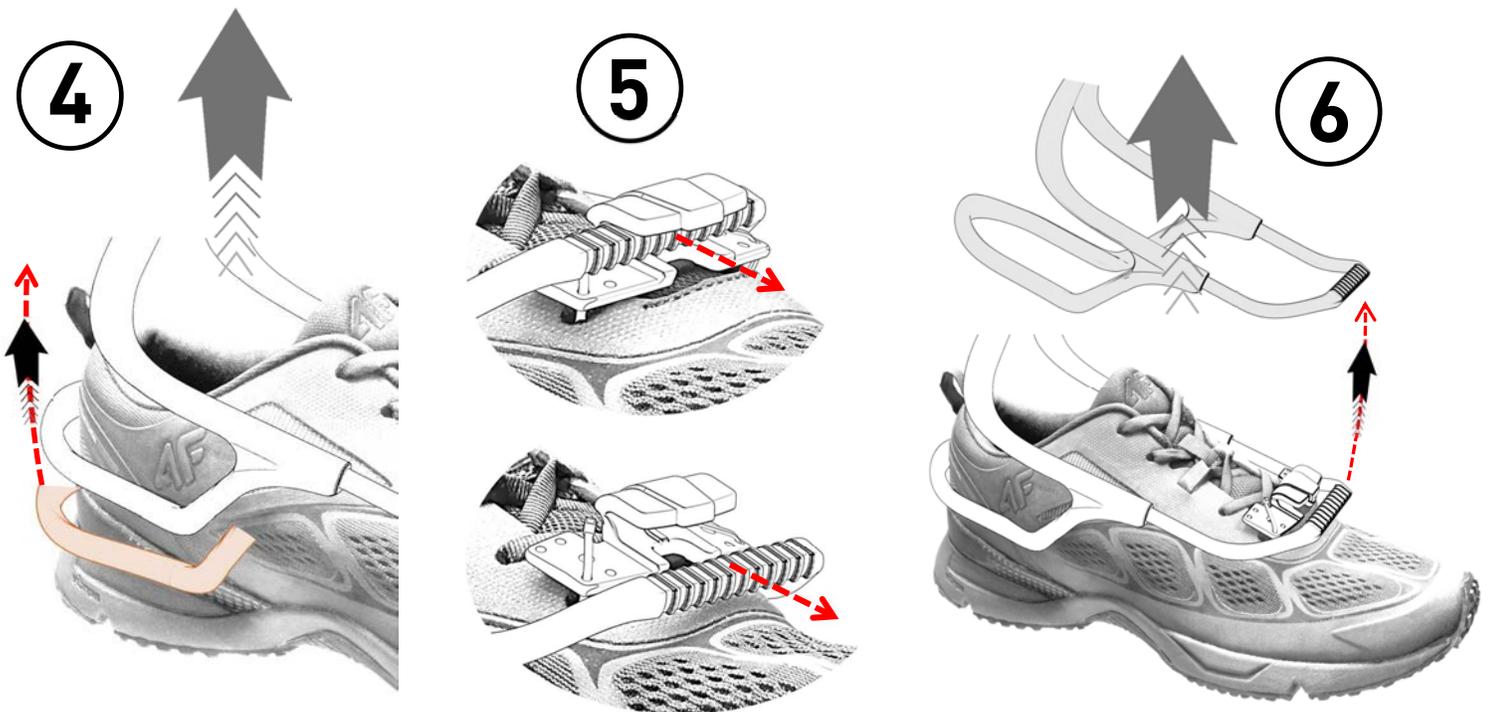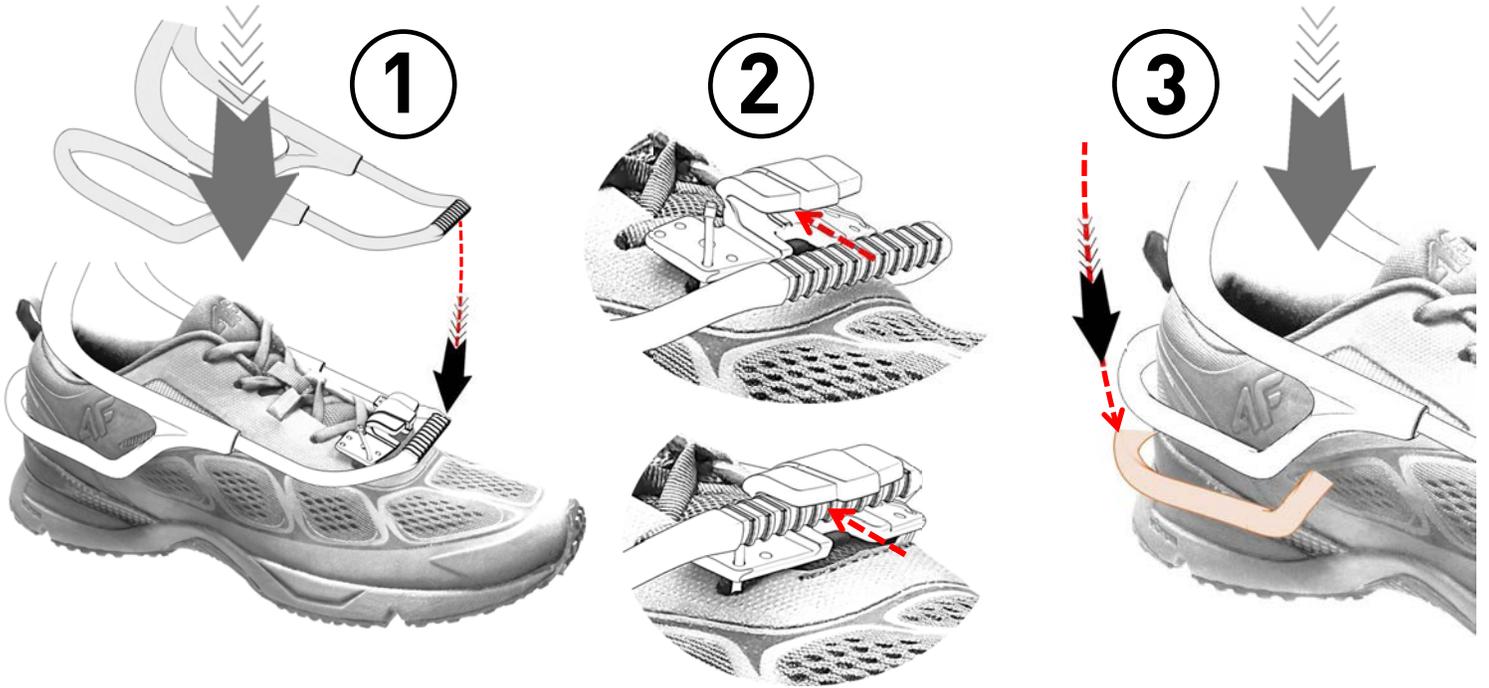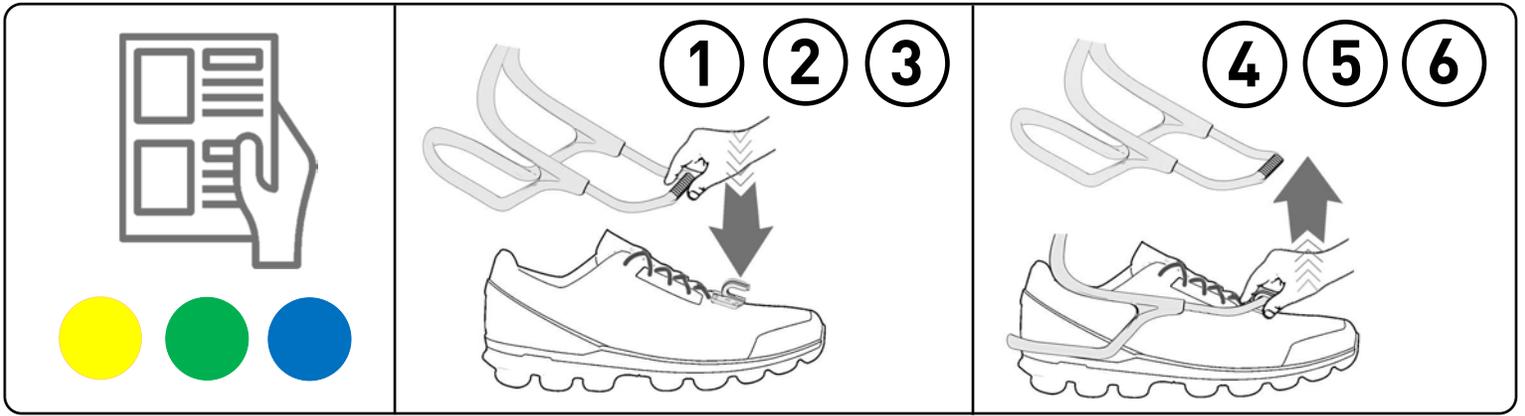1 Introduction

1.1 Copyright and intellectual property
The copyright for these assembly and use instructions is held solely by the manufacturer. Additionally, all rights are reserved, particularly with regard to patent grant or utility model registration. Violations of the above-mentioned provisions will result, where applicable, in the obligation to pay compensation for damages.
Intellectual Property – Patent numbers
US Patent No.: 8529484
CA Patent No.: 2692534
European Patent: EP2537374

1.2 Foreword
We would like to thank you for your confidence in TurboMed and the XTERN CLASSIC®. This orthosis was designed by TurboMed Orthotics Inc., located in the province of Quebec, Canada. This new-generation orthotic solution was developed specifically for people with foot drop. The orthosis CLASSIC® was developed to offer you unrivaled performance and comfort. We urge you to read this guide carefully and to address any questions and/or comments to your authorized provider before using or assembling your new orthosis. In order to ensure that the equipment works properly and safely, you must follow all of the instructions provided in this document and observe the specified usage restrictions. If, at any time, you have any concerns about the way your device is working or if you have any questions, do not hesitate to contact your authorized provider.
To contact the customer service department of TurboMed Orthotics, please visit: www.TurboMedOrthotics.com

1.3 Use
Intended use: This orthosis is intended SOLELY as an orthotic fitting for the lower limb and has been designed only for drop foot. It is important to use it in accordance with the indications.

1.4 Safety symbols used in this guide
The symbols below are used in this guide to identify the warnings and useful advice that you should observe. It is very important to read and thoroughly understand them before using our product for the first time.


Warning: this symbol means that there is a potential risk. Failure to heed a warning may result in bodily harm or injury.

Notice: recommendations or instructions should be taken seriously.

Product-related caution: concerns a potential material risk. Failure to heed a product caution may result in significant material damage.

2 Presentation of the product

Item	Description
1	MAIN SECTION
2	FRONT HARNESS "U"
3	BACK HARNESS
4	LACE CLIP
5	CALF/SHIN PAD

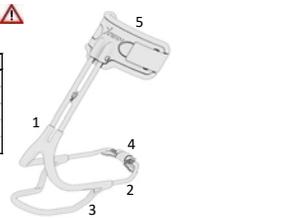3 Indications and contraindications

3.1 Indications
The orthosis is intended for people who have:
Weakness in the muscles that lift the foot, with little to no spasticity (post: back, hip or knee surgery; traumatic injury to the lower limbs; a stroke; traumatic brain injury; multiple sclerosis; Charcot-Marie-Tooth disease; peripheral neuropathy; neuromuscular atrophy, etc.)
Minor impairment of the muscles that raise the foot.
Very slight deformations of the foot capable of being corrected with the use of an insole and lateral support.
Indications for this product must always be determined by your doctor or prescribing physician.

3.2 Contraindications
The orthosis was not designed for cases of:
-moderate to severe spasticity of the lower limb;
-leg ulcers in the posterior part of the calf;
-certain balance problems;
-moderate to severe foot deformities;
-certain pathologies that require a visit to a physician to verify whether the orthosis is appropriate for the client or not.

4 The Product
(The product should be used only under the supervision of a qualified physician, MD/DO/PM)

4.1 Usage restrictions
Use of the orthosis is not recommended to prevent contractures (shortening of muscles), etc.
Avoid going near a source of high heat, such as a fire, a heating element, embers, flames, etc.
NEVER use an orthosis that belonged to someone else or an orthosis that you believe has been used.

4.2 Warning and general safety instructions (client)
Information:
-please read this document in its entirety before using the product;
-heed the safety instructions in order to avoid injury or damage to the product;
-keep this document in a safe place.

4.3 Warning and general safety instructions (professional)
Information:
-please read this document in its entirety before using the product;
-heed the safety instructions in order to avoid injury or damage to the product;
-show users how to properly use their product and inform them accurately of the safety instructions;
-keep this document in a safe place.

4.4 Caution
This product is for single patient use only. Use caution when handling the product.
Check the product regularly to see if it still works properly.
Stop using the product if there are changes in, or loss of, functionality and have it inspected by your authorized health care provider.
Observe the legal and insurance regulations related to operating motor vehicles of any type and have your driving skills tested by the competent authority.
In general, the physician will determine the daily duration of use and the total period of use.
Only qualified health care providers are authorized to perform the initial fitting and final application of the product.
Show the patient how to properly handle and care for the product.
Inform the patient that it is important to see a physician immediately if unusual changes to the body are observed (e.g. the development of a wound, worsening of pain, etc.).
Ensure that the orthosis is applied properly and that it fits correctly.
Tell the patient to check, prior to each usage, that the product works properly and that it shows no signs of wear or damage.
Direct the patient's attention to the fact that they CANNOT continue to use the product if it or one of its components is worn or damaged (ex., cracks, missing part, wiring, inappropriate shape of the orthosis, etc.).

5 Application and removal of the product

5.1 Application
1 Install the lace clip on an appropriate shoe.
2 Open the calf/shin band.
3 Install the product onto the shoe.
4 Put on the shoe and orthosis, pushing back on the orthosis.
5 Close and tie the shoe.
6 Tighten the calf/shin band so as comfortably secure the top part of the brace to the leg.
Be careful not to over tighten.

5.2 Removal
1 Open the calf/shin band.
2 Untie your shoe.
3 Remove your foot from the shoe.

5.3 Final inspection
Ask the patient to take a few steps to try out the product and to test it on slopes, ramps, while running, and on stairs. Make adjustments, if necessary.

6 Care and cleaning
Care and cleaning should be done on a regular basis. Clean the rigid parts of the product only with mild soap and water or a damp cloth. The so-called "soft" parts, such as the calf/shin pad, the band, etc., should be cleaned by hand using mild soap, then air-dried. Avoid any source of direct heat (ex., solar radiation, a radiator, fire, etc.).

7 Guarantees

7.1 Guarantee and assistance
The XTERN CLASSIC® orthosis was manufactured by TurboMed Orthotics Inc. TurboMed guarantees its orthosis for a period of two years (24 months) from the date of purchase. If a product proves to have material or manufacturing defects, TurboMed will repair it or replace it at its own expense if sufficient proof is provided—namely, a receipt (i.e. Written Proof of Delivery from your authorized provider)—and if there is proof that the date of the service request is within the guarantee period. The use of unauthorized replacement parts will cancel this guarantee. Contact your authorized provider or TurboMed for additional information in this regard.
Exclusions: the following are not covered by this guarantee:
-damage caused by improper use resulting from excessive use, negligence, fire or any other unusual accident or unauthorized modifications or repairs;
-damage caused by the use of unauthorized replacement parts;
-wear and tear including chafing, scratches, abrasions, discoloration or the fading of colors;
-the so-called "soft" parts, such as, but not limited to, the calf/shin pad, the calf/shin band, the calf/shin band pad, the plastic fasteners (tie-wraps), the various pieces of VELCRO®, etc.

7.2 Special satisfaction guarantee
We are so confident in the quality of our product that we offer to refund the cost of your orthosis if you are not satisfied (see above) within 2 months of your purchase.
TurboMed, its distributors and their authorized health care providers undertake jointly and severally to refund in full the amount paid to the client if he/she is not satisfied with the product within 2 months (2 months) of the date of purchase of the TurboMed orthosis. The client and/or authorized health care providers will be solely responsible for the shipping expenses related to sending the orthosis back to us. However, prior authorization must be requested.

8 Legal requirements
All legal requirements are subject to the national legislation of the country in question and therefore may vary depending on where the product is used.

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